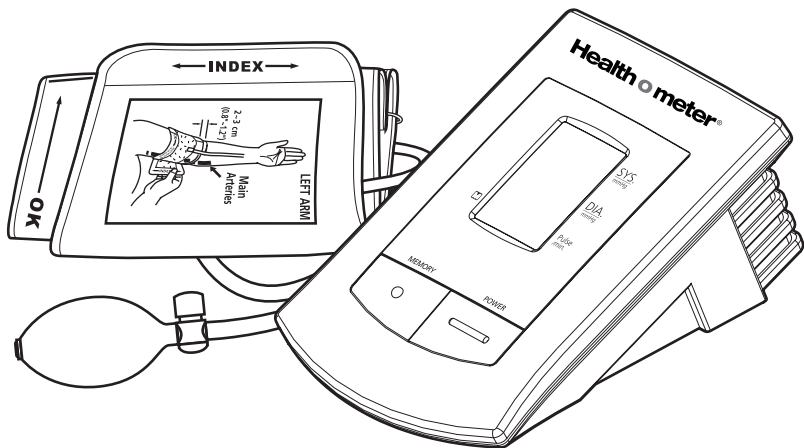
# Health o meter®

*Your Measure for Good Health™*

## UPPER ARM Blood Pressure Monitor

**Model 7630**

## IMPORTANT NOTES

Please read this entire manual carefully before using this device.

# Healthometer®

## Upper Arm Blood Pressure Monitor

### IMPORTANT SAFETY INSTRUCTIONS

**DO NOT DESTROY**

#### Features:

- **Semi-Automatic**
- **Large, Easy-To-Read Digital Display**
- **60 Automatic Memory Feature**
- **Convenient Storage Case**

*In accordance with International Standards:*

**CE** **ISO**  
0366 9001

## Table of Contents

Introduction .....	3-4
Product Component Identification .....	4
Standards of Quality .....	5
About Blood Pressure .....	5
Blood Pressure Standard .....	6
Blood Pressure Fluctuation .....	6
Display Explanations .....	7
Installing Batteries .....	7-8
HOW TO USE .....	8-12
• Helpful Tips .....	8
• Applying the Cuff .....	9-10
• Taking a Measurement .....	10-11
• Recalling Memory .....	12
Troubleshooting .....	13
Precautionary Notes .....	14
Specifications .....	15
Warranty Information .....	16

## Introduction

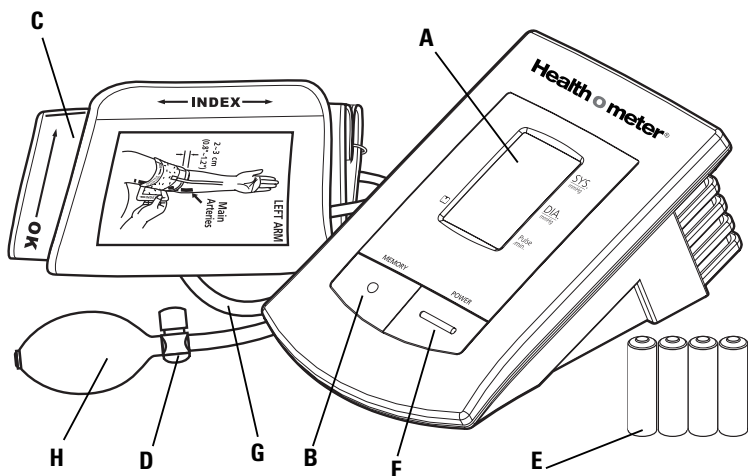
Congratulations on your purchase of the new Health o meter<sup>®</sup> Arm Blood Pressure Monitor Model 7630. This semi-automatic device is both easy to use and ideally suited for daily measurements. The large display includes systolic, diastolic and pulse measurements, all clearly shown at the completion of each reading.

In addition, you can store up to 60 measurements using the monitor's built-in memory feature, ideal for users who want to monitor and track their blood pressure on a regular basis. The Health o meter<sup>®</sup> blood pressure monitor is portable, making it ideal for both home and travel usage.

Blood pressure measurements determined with the Health o meter<sup>®</sup> blood pressure monitor are equivalent to those using the cuff/stethoscope auscultation method and are within the limits prescribed by the American National Standards Institute (ANSI). This monitor is to be used by an adult and is intended for in-home use.

Note: Please read this manual carefully before use. For specific information about your own blood pressure, contact your physician. Please be sure to keep this manual for future reference.

## Product Component Identification:

**A.** LCD Display

**B.** Memory Recall Button

**C.** Arm Cuff

**D.** Air Release Valve

**E.** 4 AA Size 1.5v Batteries (Included)

**F.** Power Button (for ON/OFF)

**G.** Air Tube and Connector

**H.** Inflating Bulb

## Standards of Quality

This Blood Pressure Monitor conforms with the American National Standards Institute/American Association of Medical Instrumentation Standards for Non-Invasive Blood Pressure Monitors. In addition, the device complies with the European regulations and bears the CE mark "CE 0366". The quality of the device has been verified and conforms to the provisions of the EC council directive 93/42/EEC dated 14 June 1993 concerning medical devices, as well as the EMC directive 89/336/EEC:

ANSI/AAMI SP10	Electronic or automated Sphygmomanometers
EN 1060-1 (Dec. 1995)	Non-invasive blood pressure measuring equipment General requirements
EN 1060-3 (Feb. 1997)	Non-invasive blood pressure measuring equipment Supplementary requirements for electromechanical blood pressure measuring systems

## ABOUT BLOOD PRESSURE

### What is Blood Pressure?

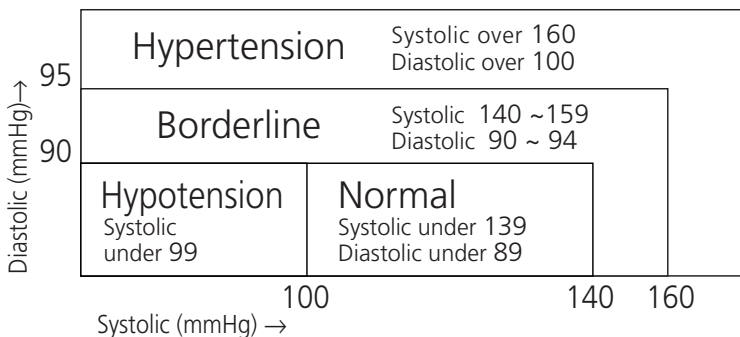
Blood pressure is the pressure exerted on the inside of the artery wall while blood flows through the artery. The pressure measured when the heart contracts and sends blood out of the heart is the "systolic" reading (higher number). The pressure measured when the heart dilates with blood flowing into the heart is called the "diastolic" (lowest number) blood pressure.

### Why Measure Your Blood Pressure?

Various health problems associated to high blood pressure are quite common with people in today's society. There is a significant correlation of high blood pressure with cardiovascular disease. This correlation has made measuring one's blood pressure a necessity for those individuals identified with high blood pressure, placing them into a high risk category pertaining to cardiovascular disease. Also, as a person ages, one's blood pressure can change over time, thus it makes good "health sense" to monitor your blood pressure on a regular basis.

## Blood Pressure Standard

The World Health Organization (WHO) has developed a blood pressure standard, according to which areas of low and high-risk blood pressure are identified. This standard, however, is a general guideline as individual blood pressure varies among different people and different age groups, etc.


It is important that you consult with your physician regularly. Your physician can tell you your optimal blood pressure range.

## Blood Pressure Fluctuation

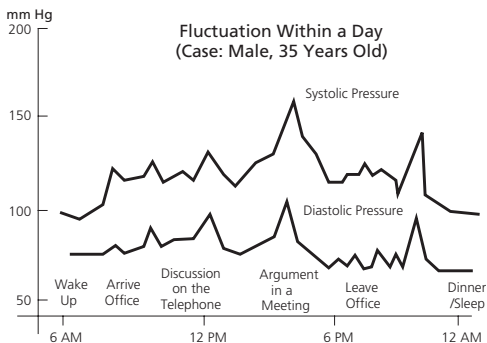
***Blood pressure fluctuates all the time!***

Blood pressure will change over the course of the month, week and even throughout the day. It is also influenced by seasonal and temperature changes.

## Display Explanations
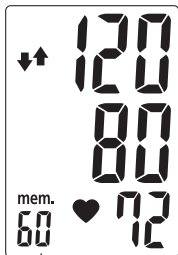
### Display:

*Systolic Pressure*

*Diastolic Pressure*

*Pulse Rate*

*No. of Memory Sequence*

### Icons:

*Appears when manual cuff inflation is needed*

*Appears when manual cuff deflation is needed*

*Appears when batteries should be replaced*

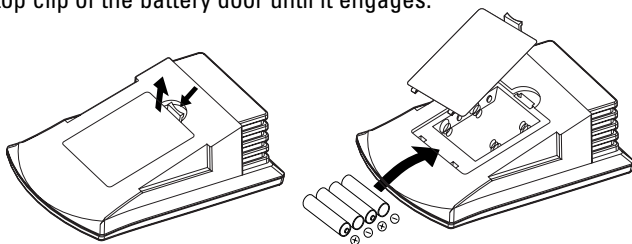
*Shows the Pulse Rate per minute*

*Occurs when a mistake was made during measurement*

## Installing Batteries

1. Press down and lift the battery cover in the direction of the arrow to open the battery compartment.
2. Install or replace 4 AA 1.5v batteries in the battery compartment according to the battery direction indicators molded on the inside of the compartment.
3. Replace the battery cover by inserting the bottom tabs first, then push in the top clip of the battery door until it engages.

## You need to replace the batteries when:

1. The “low battery” icon appears on the display.
2. The “**POWER**” button is pushed and nothing appears on the display.

Note:

Discard used Alkaline batteries as directed by your local environmental codes.

## HOW TO USE:

### Helpful Tips

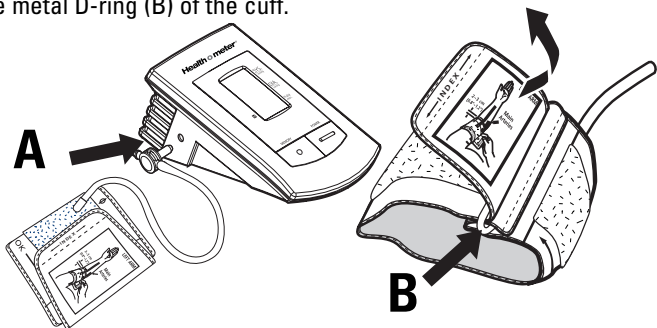
Here are a few tips to help you obtain more accurate readings:

- Do not measure your blood pressure immediately after consuming a large meal. To obtain more accurate readings, please wait one hour before measuring.
- Do not smoke or drink alcohol before measuring your blood pressure.
- You should not be physically tired or exhausted when taking a measurement.
- It is important that you relax during measurement. Try to take a 15-minute rest before taking a reading.
- Do not take measurements if you are under stress or under tension.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- **Wait about 5 minutes before taking a SECOND blood pressure measurement.**

## Applying the Arm Cuff

1. Plug the cuff connector tube (A) into the unit.
2. Unwrap the arm cuff, leaving the "Index" end of the cuff through the metal D-ring (B) of the cuff.

3. Put your left arm through the cuff loop. The "OK" range indication should be positioned at the top with the rubber tube pointing downward in the direction of your forearm. **Position the artery mark [  $\Phi$  ] over the main arteries on the inside of your arm as shown in the figure below.** The tube should be running down the center of your arm.

4. Turn your left palm upward and place the edge of the arm cuff at approximately 1 inch above your elbow. Tighten the cuff by pulling the "Index" end of the cuff. The cuff should be tensioned onto the arm to allow the insertion of two fingers between the cuff and arm. If the tension on the cuff is too tight or too loose, it may result in an error "EE" message on the display.

5. **If the Index line falls within the "OK" range, indicated at the edge of the cuff, this cuff is suitable for your use.** If the Index line falls outside of the "OK" range, you may need a larger cuff for accurate measurement. Please consult your local retailer for the purchase of a larger cuff or visit the Health o meter® website at [www.healthometer.com](http://www.healthometer.com).

Sit in a chair with your feet flat on the floor. Rest your left arm on a table so that the arm cuff is at the same level as your heart.

## Taking a Measurement

1. Press the **POWER** button (Fig. 1). All display segments will appear for approximately one second before returning to "0". **If the downward arrow "↓" flashes, this means that there is air remaining in the cuff.** (Fig. 2). **Release it by pressing the "Air Release Valve"** (Fig. 3) until "0" appears on display.

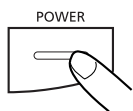
Fig. 1

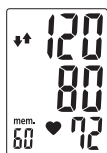
Fig. 2


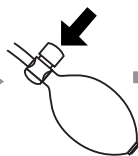
Fig. 3

The unit is ready for measurement at this point.

- Squeeze the bulb to increase pressure in the arm cuff. The display will show the pressure reading within the cuff. Continue pumping until the pressure reaches approximately **50~60 mmHg above your normal systolic pressure**.


**If the pressure in the cuff is not high enough, an upward arrow (↑) will display to remind you that you should continue pumping.**

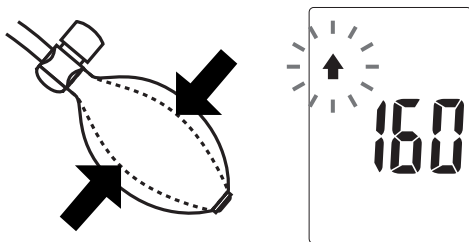- Sit still and wait for the cuff to deflate and measure your blood pressure. **It is important to remain still and quiet during measurement. Any significant movement may affect measurement results.**

- When the monitor completes the measurement, it will automatically display and store the systolic, diastolic, and pulse reading into memory. **The last 60 measurement readings will be saved into memory.**


**The display will indicate a downward arrow (↓). Keep pressing the air release valve above the bulb until all pressure in cuff is released.**

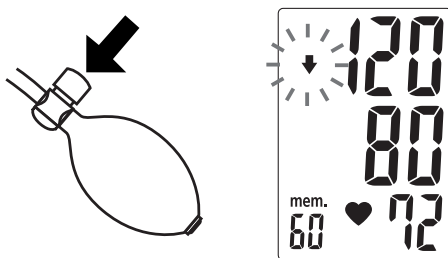- Press the **POWER** button to turn the unit off. The unit will automatically shut off after 1 minute if no buttons are pressed.

## Recalling Memory

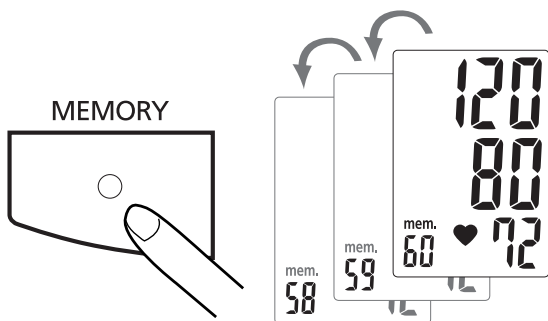1. To recall stored blood pressure readings from the memory, simply press the **MEMORY** button. The last set of readings recorded in memory will be displayed.

2. Another press of the **MEMORY** button will recall the previous set of readings.
3. All readings stored in memory will be displayed with their corresponding sequence number.

### Note:

1. The data in the memory will not be lost even when the Blood Pressure Monitor is switched off.
2. You can erase the memories stored by removing the batteries.

## Troubleshooting

If a problem should arise during use, please check the following points.

Symptom	Check Point	Correction
No display when the POWER button is pressed	Have the batteries run down?	Replace them with 4 new "AA" alkaline batteries
	Have the batteries been properly positioned in the compartment?	Re-insert the batteries in the correct position
EE mark shown on the display or the blood pressure value is displayed excessively low (or high)	Is the cuff placed correctly?	Wrap the cuff properly so that it is positioned and tensioned correctly
	Is the cuff tensioned correctly?	
	<b>Is the cuff pressure pumped enough? (50-60mmHg above normal systolic value)</b>	<b>Measure it again and pump up 50-60mmHg more than normal systolic value</b>
	Did you talk or move during the measurement?	Measure it again quietly
	Did you shake the arm with the cuff on it?	

**Note:** If the unit still does not work, please see the Warranty Information in this manual. Under no circumstance should you disassemble the unit and attempt repairs yourself.

## Precautionary Notes:

1. Your Health o meter<sup>®</sup> monitor contains high-precision components. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or sharp impacts to the main unit, and protect it from dust.
2. Clean the blood pressure monitor body and the cuff carefully with a slightly dampened soft cloth. Do not press on the unit too hard. Do not wash the cuff or use chemical cleaners on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
3. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for an extended period of time.
4. To avoid hazardous situations, the unit should not be operated by children.
5. If the unit is stored near a freezing temperature, allow it to reach room temperature before use.
6. Your Health o meter<sup>®</sup> Blood Pressure Monitor is not field serviceable. You should not use any tool to open the device nor should you attempt to make any adjustments to it. If the monitor is not working properly please contact Sunbeam Products, Inc. as outlined under the Warranty information in this manual.
7. For users diagnosed with common arrhythmia (*atrial or ventricular premature beats or atrial fibrillation*), diabetes, poor circulation of blood, kidney problems, or for users who have suffered from stroke, or for unconscious users, this device may not be suitable for use. Please consult your physician prior to use.

## Specifications:

Measurement Method	: Oscillometric
Measurement Range	: Pressure: 20 ~ 280mmHg; Pulse: 40 ~ 180 beats/minute
Pressure Sensor	: Semi Conductor
Accuracy	: Pressure: $\pm 3$ mmHg; Pulse: $\pm 5\%$ of reading
Inflation	: Manual Inflation
Deflation	: Automatic Pressure Release Valve
Memory capacity	: 60-set memory
Auto-shut-off	: 1 minute after last button operation
Operation Environment	: Temperature: 10°C – 40°C (50°F – 104°F); Humidity: 40 – 85%
Storage Environment	: Temperature: -5°C – 60°C ( 23°F – 140°F ); Humidity:10 – 95%
Power Source	: 6 Volts / Four 1.5 Volt DC (AA) Batteries
Dimensions	: 105 (L) x 160 (W) x 68 (H) mm
Weight	: 335 g (G.W.) (without Batteries)
Arm circumference	: Adult : 22 – 32 cm (8.7 – 12.6 inches) (standard) : Large Adult : 32 – 42 cm (12.6 – 16.5 inches) (optional)
K.B.	: Short time operation 2 minutes : Classification of type BF

*\*To ensure accurate measurements, it is recommended that calibration be completed every two years.*

*\*\*Specifications are subject to change without notice.*

## Warranty Information

Your Health o meter® Blood Pressure Monitor is covered under the limited Warranty offered by Sunbeam Products, Inc. ("Sunbeam"). Sunbeam warrants that for a period of five years from the date of purchase, this product will be free from defects in material and workmanship. Sunbeam, at its option, will repair or replace this product or any component of the product found to be defective during the warranty period. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. This is your exclusive warranty. This warranty is valid for the original retail purchaser from the date of initial retail purchase and is not transferable. Keep the original sales receipt. Proof of purchase is required to obtain warranty performance. Sunbeam dealers, service centers or retail stores selling Sunbeam Products do not have the right to alter, modify or in any way change the terms and conditions of this warranty.

This warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, disassembly, repair or alteration by anyone other than Sunbeam or an authorized service center. Further, the warranty does not cover Acts of God or Nature, such as fire, flood, hurricane and/or tornado.

Sunbeam shall not be liable for any incidental or consequential damages caused by the breach of any express or implied warranty. Except to the extent prohibited by applicable law, any implied warranty of merchantability or fitness for a particular purpose is limited in duration to the duration of the above warranty. Some states, provinces or jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above limitations or exclusion may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or province to province.

For more information, or to obtain warranty service, contact Sunbeam Products, Inc. at 800 672-5625 or visit the Health o meter® website at [www.healthometer.com](http://www.healthometer.com)

©2002 Sunbeam Products, Inc. All rights reserved.

Health o meter® is a registered trademark of Sunbeam Products, Inc.

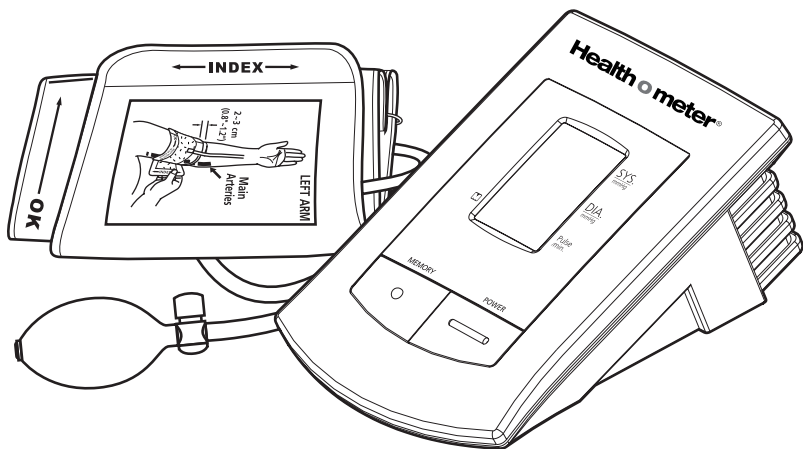
Distributed by Sunbeam Products, Inc., Boca Raton, FL 33431.

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©2002 Sunbeam Products, Inc. Todos los derechos reservados
Health o meter® es una marca registrada de Sunbeam Products, Inc.
Distribuida por Sunbeam Products, Inc., Boca Raton, FL 33431.