3M™ Attest™ Mini Auto-reader 490M

STEAM

VH202

- | | |
|------------------------------|---------------------------------|
| (en) Operator's Manual | (pt) Manual do Operador |
| (fr) Manuel de l'utilisateur | (el) Εγχειρίδιο Χειριστή |
| (de) Benutzerhandbuch | (pl) Instrukcja obsługi |
| (it) Manuale dell'operatore | (cs) Příručka pro obsluhu |
| (es) Manual del usuario | (sl) Priročnik za uporabo |
| (es) Manual del operador | (ru) Инструкция по эксплуатации |
| (nl) Bedieningshandleiding | (hr) Upute za korištenje |
| (sv) Användarhandbok | (tr) Operatör Kılavuzu |
| (da) Brugsvejledning | (zh) 操作指南 |
| (no) Brukerhåndbok | (th) คู่มือปฏิบัติงาน |
| (fi) Käyttöopas | (ko) 작업자 매뉴얼 |
| (pt) Manual do Utilizador | |

**Read and understand all safety information
before using this product.**

TABLE OF CONTENTS

PREFACE	2
INTENDED USE.....	2
INDICATIONS FOR USE.....	2
WARRANTY	2
SAFETY AND PRECAUTIONS.....	2
HEALTH AND SAFETY INFORMATION	3
EXPLANATION OF PRODUCT AND PACKAGING LABEL SYMBOLS.....	3
SPECIFICATIONS.....	4
INSTRUCTIONS FOR USE.....	4
FEATURES OF THE 3M™ ATTEST™ MINI AUTO-READER 490M.....	7
MAINTENANCE, CARE AND CLEANING	8
TROUBLESHOOTING GUIDE	9
OPTIONAL 3M™ ATTEST™ CONNECT AUTO-READER APPLICATION.....	10
TROUBLESHOOTING - 3M™ ATTEST™ CONNECT	12
CUSTOMER SUPPORT.....	12

PREFACE

Content Disclaimers

Pictorial Disclaimer

Sample printouts, graphics, displays and screens are for information and illustration purposes only and shall not be used for clinical or maintenance evaluations. Data shown in sample printouts and screens do not reflect actual test results.

Hardware Disclaimer

The 3M™ Attest™ Mini Auto-reader 490M hardware specifications are subject to change. The system images, hardware components and hardware specifications included in the manual may not match the system installed. Any changes or modifications to the authorized system installation have been verified as compatible with the functionality outlined in this document.

INTENDED USE

The 3M™ Attest™ Rapid Biological Indicator System provides rapid, reliable monitoring of steam and vaporized hydrogen peroxide sterilization processes performed in health care facilities. The system is comprised of biological indicators, challenge packs, and incubator/readers.

The 3M™ Attest™ Mini Auto-reader 490M is designed to incubate and automatically read 3M™ Attest™ Biological Indicators 1295, 1491, and 1492V at 60°C for a final negative fluorescence reading at 24 minutes. The 3M™ Attest™ Mini Auto-reader 490M is also designed to allow for further incubation of 3M™ Attest™ Super Rapid Readout Biological Indicators, catalog numbers 1491 and 1492V, for a final negative, visual pH color change result at 24 hours and 48 hours, respectively. This further incubation is optional and not intended for routine use.

A positive fluorescent result or a visual pH color change indicate a sterilization process failure. However, due to the high sensitivity of the fluorescent result, there is no advantage to continuing to incubate processed 3M™ Attest™ Biological Indicators once the fluorescent result has been documented.

Other biological indicators (BIs) are not compatible with this device and cannot be used.

INDICATIONS FOR USE

The 3M™ Attest™ Mini Auto-reader 490M is designed to incubate and automatically read 3M™ Attest™ Rapid Readout Biological Indicators 1295 and 3M™ Attest™ Super Rapid Readout Biological Indicators, catalog numbers 1491 and 1492V, at 60°C for a final fluorescent result at 24 minutes.

The indications for use for Attest™ Biological Indicators are provided in the product's Instructions for Use.

WARRANTY

In the US, the 3M™ Attest™ Mini Auto-reader 490M has a one-year limited product warranty. The US warranty, remedy and limitations are described on the product webpage at Go.3M.com/490M. For countries outside of the US, any warranty is established by the subsidiary with responsibility for sales of the 3M™ Attest™ Mini Auto-reader 490M.

Register your device at Go.3M.com/490M.

SAFETY AND PRECAUTIONS

The 3M™ Attest™ Mini Auto-reader 490M and its related devices and accessories are designed to provide safe and reliable service when used according to the instructions provided. Please read, understand, and follow all safety information contained in the instructions for use included with the 3M™ Attest™ Mini Auto-reader 490M and the associated Attest™ biological indicators prior to use. Use this equipment only for the purpose described in the Operator's Manual. **Retain these instructions for future reference.**

The unit is designed to be used only with the Power Supply module and USB cable supplied by 3M. If this product is used in a manner not specified by 3M, the protection provided by the product may be impaired.

Explanation of Signal Word Consequences

WARNING: Indicates a hazardous situation, which, if not avoided, could result in death or serious injury.

CAUTION: Indicates a hazardous situation, which, if not avoided, could result in minor or moderate injury.

Warnings and Safety Precautions

The following warnings and precautions should be followed to avoid unsafe actions that could result in personal injury or damage to the instrument.

WARNING: To reduce the risk associated with hazardous voltage

Use indoors only.

Do not use the equipment if it is not working properly or if it has suffered any damage.

Use only the power supply specified for this product and certified for the country of use.

CAUTION: To reduce the risk of injury or instrument damage

Do not spill liquid onto or into the instrument. Do not immerse the unit in liquid.

Always unplug the 3M™ Attest™ Mini Auto-reader 490M and allow to cool before cleaning.

Clean external surfaces using only the instructions provided by the manufacturer.

Do not open the instrument housing – there are no user-serviceable parts.

Wear safety glasses and gloves when activating the 3M™ Attest™ Rapid Readout Biological Indicator 1295.

Allow steam-processed 3M™ Attest™ Biological Indicators to cool for the recommended time period before activating. Activating or excessive handling of the biological indicator before cooling may cause the glass media ampoule to burst.

Wear safety glasses when activating 3M™ Attest™ Super Rapid Biological Indicators.

WARNING: To reduce the risk associated with incorrect results

Instrument is to be used by operators familiar with the device, device functionality, and Operator's Manual.

Do not place the instrument in environment exposed to sunlight or strong incandescent light.

Do not place instrument close to any device that emits a strong electro-magnetic field.

Do not use on a vibrating surface.

Activate and incubate 3M™ Attest™ Rapid Readout Biological Indicators 1295 within 1 hour of the completion of the sterilization cycle.

Do not incubate a 3M™ Attest™ Biological Indicator if, after processing and before BI activation, it is observed to have a broken media ampoule. Retest the sterilizer with a new biological indicator.

Allow steam-processed 3M™ Attest™ Biological Indicators to cool for the recommended time period before activating.

Do not remove or change placement of 3M™ Attest™ Biological Indicator once it is placed into a well.

Do not remove the 3M™ Attest™ Biological Indicator from the incubation well until the (+) or (-) symbol on the display indicates the test is complete.

Do not re-incubate 3M™ Attest™ Biological Indicators for which the Auto-reader has already determined a result.

HEALTH AND SAFETY INFORMATION

Device Safety Compliance

The 3M™ Attest™ Mini Auto-reader 490M complies with the following standards as demonstrated by the CB Scheme Certificate and test report issued by UL, LLC (UL):

- IEC/EN 61010-1:2010. Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 1: General requirements
- IEC/EN 61010-2-010:2014. Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 2-010: Particular requirements for laboratory equipment for the heating of materials.

The 3M™ Attest™ Mini Auto-reader 490M is UL Listed and carries the UL mark with adjacent indicators "C" and "US" based on compliance to the standards UL 61010-1, CAN/CSA 22.2 No. 61010-1 and CAN/CSA 22.2 No. 61010-2-010.

The 3M™ Attest™ Mini Auto-reader 490M complies with the CE mark related to the Low Voltage Directive (LVD) 2014/35/EU as confirmed in the Declaration of Conformity.

The 3M™ Attest™ Mini Auto-reader 490M complies with the RoHS Directive, Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

The 3M™ Attest™ Mini Auto-reader 490M complies with China RoHS II requirements.

The 3M™ Attest™ Mini Auto-reader 490M complies with the WEEE Directive, Directive 2012/19/EU of the European Parliament and of the Council of 04 July 2012 on waste electrical and electronic equipment (WEEE).

EMC Compliance

The 3M™ Attest™ Mini Auto-reader 490M complies with the following EMC standards as confirmed in the Declaration of Conformity generated by 3M:

- IEC 61326-1:2012/EN 61326:2013 Electrical equipment for measurement, control and laboratory use - EMC requirements - Part 1: General requirements
- EMC requirements of the CE mark EMC Directive 2014/30/EU.

The 3M™ Attest™ Mini Auto-reader 490M complies with the Australian and New Zealand electrical safety and electromagnetic compatibility requirements as confirmed in the Supplier's Declaration of Conformity that is linked to the Australian/New Zealand Regulatory Compliance Mark (RCM).

CAN ICES-3 (B)/NMB-3(B)

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Modifications to this device shall not be made without the written consent of 3M Company. Unauthorized modifications may void the authority granted under Federal Communication Rules permitting the operation of this device.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

EXPLANATION OF PRODUCT AND PACKAGING LABEL SYMBOLS

Symbol Glossary

Symbol Title	Symbol	Description and Reference
Manufacturer		Indicates the medical device manufacturer as defined in Medical Device Regulation (EU) 2017/745 formerly EU Directive 93/42/EEC. Source: ISO 15223, 5.1.1
Authorized Representative in European Community		Indicates the authorized representative in the European Community. Source: ISO 15223, 5.1.2, 2014/35/EU, and/or 2014/30/EU
Date of Manufacture		Indicates the date when the medical device was manufactured. Source: ISO 15223, 5.1.3
Catalogue number		Indicates the manufacturer's catalogue number so that the medical device can be identified. Source : ISO 15223, 5.1.6
Serial number		Indicates the manufacturer's serial number so that a specific medical device can be identified. Source: ISO 15223, 5.1.7
Caution		Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself. Source: ISO 15223, 5.4.4
CE Mark		Indicates conformity to all applicable European Union Medical Device Regulations and Directives.
UL Listed		Indicates product was evaluated and Listed by UL for the USA and Canada.
Regulatory Compliance Mark		Indicates product complies with applicable Australian regulatory requirements. Source: AS/NZS 4417.1:2012
China RoHS II symbol		Indicates the product's environmental-friendly use period in years. Source: SJ/T 11364-2014
Direct current		To indicate on the rating plate that the equipment is suitable for direct current only or to identify relevant terminals. Source: IEC 60417-5031
Recycle electronic equipment		DO NOT throw this unit into a municipal trash bin when this unit has reached the end of its lifetime. Please recycle. Source: Directive 2012/19/EC on waste electrical and electronic equipment (WEEE)

For more information see, HCBGRegulatory.3M.com

SPECIFICATIONS

Power Supply Specifications

Power supply	Operating Condition
Voltage Range	100-240
Frequency	50/60
Current	0.6

Units
Volts AC
Hertz
Amps

Environmental Operating Conditions

Environmental Condition	Operating Condition
Altitude	3000 (max)
Operating Temperature	16 – 35
Storage Temperature	-10 to 50
Operating Relative Humidity	20-80 (non-condensing)
Voltage Range	12
Frequency	DC
Current	1.2
Installation/Over Voltage	Category II
Pollution Degree	2

Units
Meters
°C
°C
%
Volts DC
Amps

Unit Dimensions

Width - 15.5 cm
 Height - 8.25 cm
 Depth - 5.25 cm

INSTRUCTIONS FOR USE – Initial Set-up

Choose Location

Place 3M™ Attest™ Mini Auto-reader 490M on a firm level surface away from sunlight and incandescent light.

Select & Install Correct Power Plug

The 3M™ Attest™ Mini Auto-reader 490M power supply comes with multiple blade configurations. Install the blade configuration appropriate for your country into the power supply. Connect the power supply to the power receptacle located on the back of the 3M™ Attest™ Mini Auto-reader 490M and then plug into an appropriate AC grounded outlet.

The power supply plug serves as the electrical supply source disconnecting device. Do not position any equipment around the power supply which makes it difficult to unplug the power supply.

The bootloader (B X.X.X) and firmware (V X.X.X) versions are briefly sequentially displayed when the 3M™ Attest™ Mini Auto-reader 490M is powered on.

WARNING:

- Do not place the instrument in environment exposed to sunlight or strong incandescent light.
- Do not place instrument close to any device that emits a strong electro-magnetic field.
- Do not use on a vibrating surface.

Wait 30 Minutes Before Using

Once plugged in, allow a 30 minute warm-up period before placing biological indicators into the incubation wells. The C1 caution code will disappear when proper incubation temperature is reached.



Special Considerations

It is recommended that the unit be left on to eliminate warm-up periods.
 The unit may be unplugged when it is anticipated that it will not be used for an extended period.

WARNING: Use only the power supply specified for this product and certified for the country of use.

INSTRUCTIONS FOR USE – Wells, Button, Display & Connections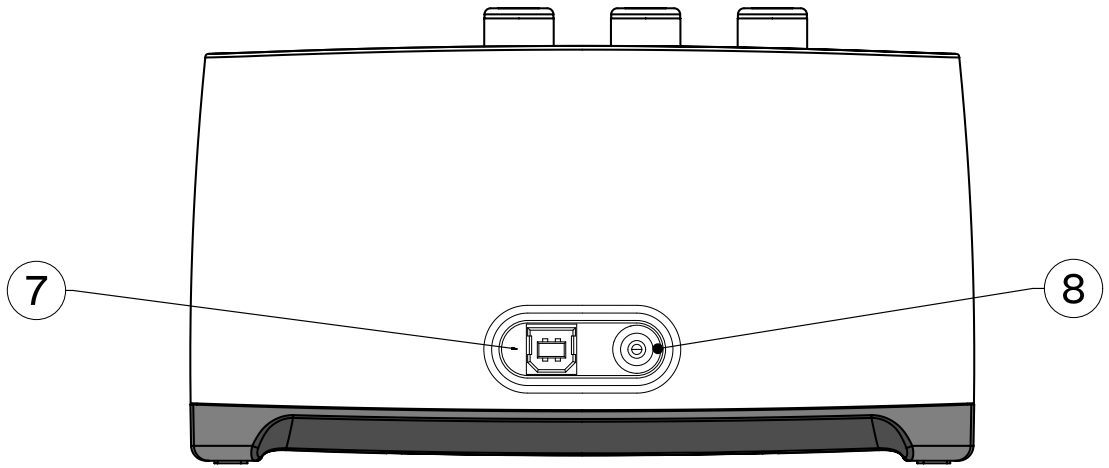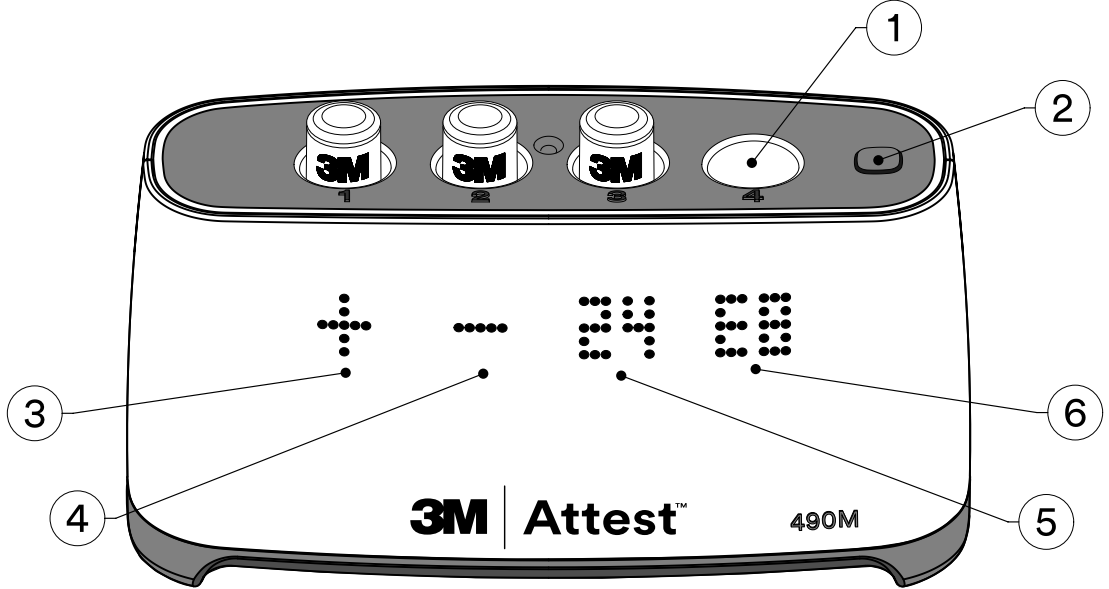

WELLS

1. Incubation wells – the 3M™ Attest™ Mini Auto-reader 490M has four incubation wells.

Button

2. Touch button (Alarm OFF)

Display

- 3. Example of Positive (+) BI result for well #1
- 4. Example of Negative (-) BI result for well #2
- 5. Example of remaining incubation time for well #3
- 6. Example of error code for well #4

Rear View

- 7. USB cable port – connect “B” end of supplied USB cable here to use 3M™ Attest™ Connect
- 8. Power receptacle – connect power supply AC adaptor here

INSTRUCTIONS FOR USE – Readout Procedure

Instructions for Biological Indicator (BI) Activation and Incubation

CAUTION: Allow steam-processed 3M™ Attest™ Biological Indicators to cool for the recommended time period before activating. Activating or excessive handling of the biological indicator before cooling may cause the glass ampoule to burst.

CAUTION: Wear safety glasses when activating 3M™ Attest™ Super Rapid Biological Indicators.

CAUTION: Wear safety glasses and gloves when activating the 3M™ Attest™ Rapid Readout Biological Indicator 1295.

WARNING: To reduce the risk associated with incorrect results


Do not incubate a 3M™ Attest™ Biological Indicator if, after processing and before BI activation, it is observed to have a broken media ampoule. Retest the sterilizer with a new biological indicator.

Activate and incubate 3M™ Attest™ Rapid Readout Biological Indicators 1295 within 1 hour of the completion of the sterilization cycle.


Allow steam-processed 3M™ Attest™ Biological Indicators to cool for the recommended time period before activating.

Do not re-incubate 3M™ Attest™ Biological Indicators for which the Auto-reader has already determined a result.


Follow this procedure for every 3M™ Attest™ Biological Indicator (BI) to be read: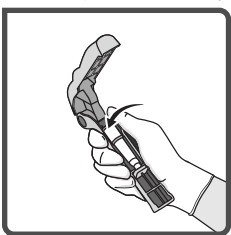


While wearing safety glasses (and gloves for the 1295 BI), **PLACE** BI in Attest™ Biological Indicator Activator.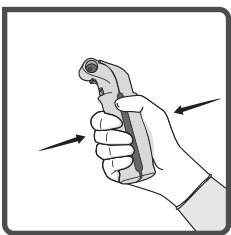

CLOSE and **SQUEEZE** Attest™ Biological Indicator Activator to close BI cap and crush media ampoule.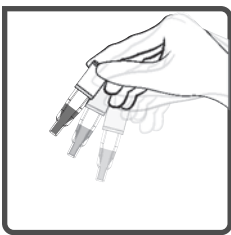

Remove BI, visually **VERIFY** the glass ampoule (containing growth media) has been crushed, and **FLICK** BI to distribute purple growth media to the growth chamber at the bottom of the vial.

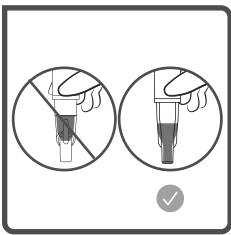INSPECT to verify media has transferred to growth chamber at the bottom of the BI vial.

Orient flat side of activated BI towards front of 3M™ Attest™ Mini Auto-reader 490M and **PLACE** the BI into any incubation well. Observe 3M™ Attest™ Mini Auto-reader 490M display to confirm correct placement of the BI. If the BI is correctly seated, the 3M™ Attest™ Mini Auto-reader 490M will display ●●●, ●●, ●, and then remaining minutes of incubation directly below the incubating BI

RECORD BI result.

WARNING: Do not remove the biological indicator from the incubation well until the (+) or (-) symbol on the display indicates the test is complete.

INSTRUCTIONS FOR USE – Additional Information

BI Placement

All incubation wells in the 3M™ Attest™ Mini Auto-reader 490M are programmed to provide a final fluorescent result within 24 minutes of incubation and may be used to incubate 3M™ Attest™ Super Rapid Biological Indicators 1491 and 1492V and the 3M™ Attest™ Rapid Readout Biological Indicator 1295.

Remaining incubation time, in minutes, will be displayed on the front of the unit to indicate the 3M™ Attest™ Biological Indicator is incubating in the reader well and should not be touched. The time remaining display indicates the results are not available and incubation is in progress. **Do not remove 3M™ Attest™ Biological Indicators until the (+) or (-) symbol on the display indicates the test is complete.**

Note: If a 3M™ Attest™ Biological Indicator is inadvertently removed before the incubation period is complete, an alarm will sound, and a C2 caution code will be displayed under that well. **You must return the 3M™ Attest™ Biological Indicator to the incubation well within 10 seconds or the results will be lost.**

WARNING: Do not remove or change placement of 3M™ Attest™ Biological Indicator once it is placed into a well. Doing so may result in loss of results and the biological indicator test may be invalid.

INSTRUCTIONS FOR USE

Positive Control

Activate and incubate at least one non-processed 3M™ Attest™ Biological Indicator to use as the positive control each day that a processed 3M™ Attest™ Biological Indicator is incubated and read by the unit. Write a “C” (for “control”) and the date on the label. The positive control can be placed in any incubation well. A positive control should be run for each catalog number and have the same lot number as the processed 3M™ Attest™ Biological Indicator.

The positive control BI helps confirm:

- correct incubation temperatures are met
- viability of spores has not been altered due to improper storage temperature, humidity, or proximity to chemicals
- capability of media to promote rapid growth, and
- proper functioning of 3M™ Attest™ Mini Auto-reader 490M components.

INSTRUCTIONS FOR USE – Interpretation of Results

Fluorescent Results

As soon as a positive 3M™ Attest™ Biological Indicator result is detected, a flashing (+) symbol will be displayed directly under the incubating BI and an alarm will sound (if the sound feature is enabled). Tap the touch button to acknowledge the result, stop the flashing display, and silence the alarm (if enabled). At the end of the 24-minute incubation time, if a negative 3M™ Attest™ Biological Indicator result is detected, a (-) symbol will be displayed directly under the incubating BI. The final negative fluorescent result (-) for a processed 3M™ Attest™ Biological Indicator after 24 minutes of incubation indicates an acceptable sterilization process.

FEATURES OF THE 3M™ ATTEST™ MINI AUTO-READER 490M

Audible Alarm

An audible alarm will sound and a flashing (+) symbol will be displayed directly under the incubating BI when a positive result is detected. An alarm will also sound if an incubating BI is removed from the 3M™ Attest™ Mini Auto-reader 490M incubation well before a result has been determined. Data will be lost if the BI is not replaced in the incubation well within 10 seconds.

The audible alarm for positive results can be silenced for a particular 3M™ Attest™ Biological Indicator. This is accomplished by pressing the touch button after the alarm has sounded. This will turn off the alarm for that BI only. Any positive 3M™ Attest™ Biological Indicator results subsequently detected in other incubation wells will result in the alarm sounding. Removing the BI reactivates the alarm for that well. Pressing the touch button to silence the alarm for a positive BI does not disable the alarm for error or caution code alarm warnings. The default factory setting is audible alarm enabled. The alarm can be disabled in the 3M™ Attest™ Connect Settings page.

After either the + or – symbol is displayed, and the result is recorded, the 3M™ Attest™ Biological Indicator can be removed from the well. The result (+ or -) is displayed until the BI is removed from the incubation well. The result clears ten seconds after the BI has been removed. Do not re-incubate BIs as the result will not be valid.

The unprocessed control BI must provide a positive fluorescent result (+). Processed BI results are not valid until the daily positive control BI reads fluorescent positive (+).

With a processed 3M™ Attest™ Biological Indicator, a positive (+) result indicates a sterilization process failure. Act on any positive test results immediately. Determine the cause of the positive 3M™ Attest™ Biological Indicator and retest the sterilizer following facility policies and procedures.

Optional Visual pH Color Change Result - 3M™ Attest™ Super Rapid Readout Biological Indicators 1491 and 1492V

3M™ Attest™ Super Rapid Readout Biological Indicators may be further incubated for a visual pH color change. This optional further incubation is not intended for routine use and may be used for special studies. Due to the high sensitivity of the fluorescent result, there is no advantage to incubating the 3M™ Attest™ Super Rapid Readout Biological Indicators 1491 and 1492V beyond 24 minutes. After activation and during incubation, the white Nonwoven Material in the biological indicator will absorb bromocresol purple indicator, the pH-sensitive indicator dye in the growth media, and appear blue. In the case of a positive control BI, a yellow color change of the growth media and/or Nonwoven Material will appear within 24 hours for 1491 BIs and within 48 hours for 1492V BIs. Any observation of a yellow color within the vial indicates a positive result. In the case of a processed BI, a media and/or Nonwoven material color change from purple to yellow indicates a steam sterilization process failure. A negative pH color change result, i.e. media and Nonwoven Material remain purple/blue, can be assessed at 24 hours for 1491 BIs and at 48 hours for 1492V BIs.

Optional Visual pH Color Change Result - 3M™ Attest™ Rapid Readout Biological Indicator 1295

3M™ Attest™ Rapid Readout Biological Indicators 1295 may be further incubated for a visual pH color change. Due to the high sensitivity of the fluorescent result, there is no advantage to continuing to incubate 3M™ Attest™ Rapid Readout Biological Indicators 1295 after the fluorescent result has been determined and documented. This optional further incubation is not intended for routine use but may be used for special studies.

In the case of a positive control BI, a yellow color change of the growth media will appear within 24 hours. Any observation of a yellow color within the vial indicates a positive result. The pH color change result can be recorded, and the BI discarded as soon as the yellow color is observed.

If a visual pH color change result is desired for processed 3M™ Attest™ Rapid Readout Biological Indicators 1295, the use of a humidified incubator is recommended to avoid media dry-out. Transfer the processed 1295 BI to a humidified incubator operating at 60°C after the fluorescent result has been determined by the 3M™ Attest™ Mini Auto-reader 490M. A media color change from purple to yellow indicates a sterilization process failure. A negative pH color change result, i.e., media remains purple, can be assessed at 7 days.

INSTRUCTIONS FOR USE – Disposal

Instrument contains a lithium battery. Dispose of instrument in accordance with federal, state, and local requirements.

Serial Number, Firmware Version, and Temperature Status

To check the serial number, firmware version, or heater block temperature, press and hold the touch button. The unit's serial number, firmware version, and incubator block temperature will sequentially appear on the Display. The 3M™ Attest™ Mini Auto-reader 490M contains an internal temperature self-diagnostic program. Should the temperature fall outside the specification (60.0°C +/- 2.0°C), an error code will be displayed.

Power Disruptions

When electrical power is restored following a power disruption, the 3M™ Attest™ Mini Auto-reader 490M checks the heater block temperature.

- A C1 caution code is displayed under all unoccupied incubation wells if the temperature is below specification. The C1 caution code will clear when the temperature reaches specification (60.0°C +/- 2.0°C).
- For any incubating BIs for which the result was determined prior to the power disruption, the result is displayed when power is restored provided the BI remains in the well.
- For any incubating BIs for which the result was not determined prior to the power disruption:
 - If the heater block temperature is within specification, readings resume as indicated by the remaining incubation time under the BIs on the Display.
 - After short power disruptions, if the heater block temperature is above 50°C, but not within specification, a C3 caution code is displayed under affected wells and readings are not taken. Readings resume, as indicated by the remaining incubation time on the Display, when the temperature reaches specification.
 - If the heater block temperature is below 50°C, the E5 error code is displayed below affected BIs. A result will not be determined, and the load/sterilizer should be tested with a new BI per facility protocol.

Optional 3M™ Attest™ Connect

If desired, the 3M™ Attest™ Mini Auto-reader 490M may be connected to a Personal Computer (PC) via the supplied USB cable for additional functionality. Downloading and installing the free 3M™ Attest™ Connect Auto-reader application allows users to:

- View current status of BIs in the 3M™ Attest™ Mini Auto-reader 490M on a PC screen
- View and export previous BI records to create print-outs, and/or save documentation as an electronic file.*
- Interface with compatible Instrument Tracking software systems
- Create diagnostic files for remote customer service support and trouble-shooting
- Configure 3M™ Attest™ Mini Auto-reader 490M settings
- Enable future upgrades of the 3M™ Attest™ Mini Auto-reader 490M firmware

See the **OPTIONAL 3M™ Attest™ Connect Auto-reader Application** section of this manual for more details.

* Refer to your facility's record keeping policy and procedure. The user is solely responsible for the integrity and security of the data on their PC.

MAINTENANCE, CARE AND CLEANING

The 3M™ Attest™ Mini Auto-reader 490M does not contain any user-serviceable parts or require any routine maintenance or calibration. Any malfunctions will be indicated by an error code. Refer to the Troubleshooting Guide section for error code explanations.

Clean the exterior surface of the 3M™ Attest™ Mini Auto-reader 490M by wiping it with a damp cloth moistened with a mild detergent. Do not attempt to clean interior components.

	CAUTION: Always unplug the 3M™ Attest™ Mini Auto-reader 490M and allow to cool before cleaning.
Do not spill liquid onto or into the instrument. Do not immerse the unit in liquid.	
Clean external surfaces using only the instructions provided by the manufacturer.	
Do not open the instrument housing – there are no user-serviceable parts.	

TROUBLESHOOTING GUIDE

The Display will indicate detected malfunctions by displaying a caution or error code under affected BI wells. The audible alarm, if enabled, sounds and the Display flashes to alert the user to all caution (except C1 and C3) and error codes. Press the touch button to stop the flashing and silence the audible alarm (if enabled). Refer to the following:

Code	Problem	Solution
C1	Unit Warm-up Caution Incubation block not at temperature.	Allow a 30-minute warm-up after 3M™ Attest™ Mini Auto-reader 490M is plugged in. (Note: 3M™ Attest™ BI readings are disabled until incubator block reaches 60°C +/- 2°C).
C2	BI Removed from Well Caution 3M™ Attest™ BI has been removed before incubation is complete.	Replace 3M™ Attest™ BI in proper well within 10 seconds to prevent loss of collected data.
C3	Auto-reader heater block not within temperature specification following a short power disruption.	No action is required, system will recover. C3 is displayed under incubating BIs for which result has not yet been determined. Readings are suspended until heater block temperature is within specification.
C6	Purple growth media not detected in biological indicator growth chamber.	Readings will not begin until C6 caution code is cleared. To clear caution code, remove BI and inspect to verify cap is closed, media ampoule is crushed, and media has filled growth chamber at the bottom of the vial. If necessary, repeat BI activation steps, FLICKING BI to distribute growth media to the growth chamber, and return BI to incubation well.
E0	Calibration error detected.	Contact Customer Support.
E1	Temperature Control Loss Error occurs if system is no longer able to control the temperature of the incubator heater block to 60°C +/- 2°C.	Unplug the unit and plug it back in to clear error. If error doesn't clear, unplug the unit and contact Customer Support.
E3	Memory Failure Error 3M™ Attest™ Mini Auto-reader 490M unable to save/read data from memory.	Unplug the unit and plug it back in to clear error. If error doesn't clear, contact Customer Support.
E4	BI Inserted during unit warm-up	Remove BI. Wait for C1 Caution Code to clear before incubating BI.
E5	Power Failure Error	Ensure power supply connection to Auto-reader is secure. Affected BI result(s) inconclusive as power to Auto-reader was off for too long to provide valid result for affected BI(s). Retest load/sterilizer with a new BI per facility policy.
E8	Optical Detection System Fault Detected This error occurs for a specific well if the system detects that a UV LED or optics detection circuit is not working properly.	Unplug the unit and plug it back in to clear error. If error doesn't clear, contact Customer Support. Unaffected well(s) may be used to incubate BIs.
Other	Unwanted alarm sounds when positive BI detected.	Disable audible alarm. See instructions in 3M™ Attest™ Connect Settings page section.
Other	Alarm Sounds 3M™ Attest™ Biological Indicator has been removed from incubation/reader well prematurely.	Replace 3M™ Attest™ Biological Indicator in incubation/reader well within 10 seconds to prevent loss of data collected.
Other	Display does not function properly.	Unplug the unit and plug it back in. If problem persists, contact Customer Support.
Other	Negative Result for Positive Control BI BI not properly activated. Media did not contact spore carrier. Control BI was sterilized.	Ensure BI cap is closed and glass ampoule is crushed. Flick BI until media flows to spore chamber at bottom of BI. Check process indicator on BI cap. Use a non-sterilized BI as a control.
Other	Unexplained Positive Result for Processed BI Tape or label may impact result.	Do not place tape or labels on BI.
Other	Display does not acknowledge BI	Check BI orientation and ensure BI is properly seated in incubation well. Only use BIs that are compatible with this device. Contact Customer Support.

OPTIONAL 3M™ ATTEST™ CONNECT AUTO-READER APPLICATION

Use of 3M™ Attest™ Connect provides additional functionality as described in the Features section of this manual.

Personal Computer (PC) Requirements

3M™ Attest™ Connect is designed to be used with PCs with internet access running either the Windows 7 or Windows 10 operating systems. The 3M™ Attest™ Connect application will run in the first supported language specified in the PC's language settings. For best results, a minimum PC display resolution of 1440 X 900 is recommended.

Getting Started

Open a web browser on the PC and navigate to Go.3M.com/490M. Click on the REGISTER PRODUCT button to begin the registration process. When prompted, click on the Download Attest Connect for Windows button.


Double click on the **Setup3MAttestConnect.exe** file and follow the prompts on the screen to review and accept the End User License Agreement (EULA) and install 3M™ Attest™ Connect. 3M™ Attest™ Connect will automatically start.

Next, connect a 3M™ Attest™ Mini Auto-reader 490M to the PC using the supplied USB cable. The first time an Auto-reader is connected, you will be presented with a "Sign In" form. Sign in using the same email address that was entered in the registration process. The sign in process links the installation of 3M™ Attest™ Connect on your PC with the registration information created in the registration website.

Once the sign in form completes, the **Home** page will appear on the PC screen. 3M™ Attest™ Connect has four main pages: Home, History, Settings, and BI Details.

HOME PAGE

As depicted below, when the 3M™ Attest™ Auto-reader 490M is connected to the PC, the Home page provides a real-time, virtual image of the Auto-reader along with other information and functionality.

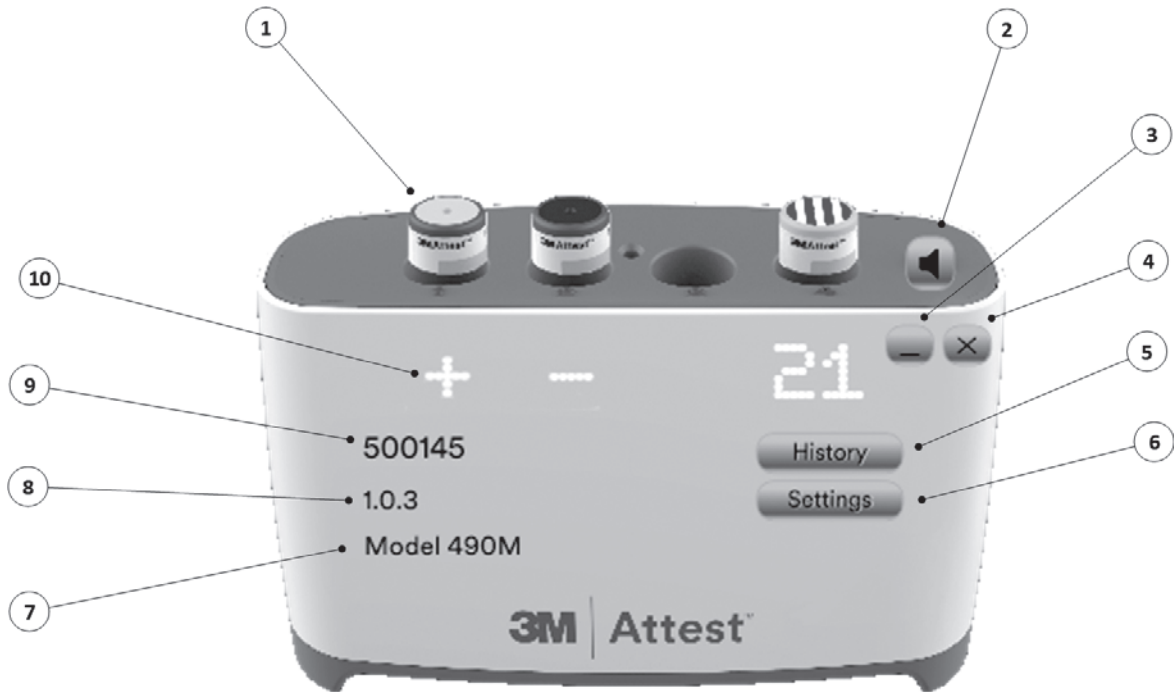- | | |
|--|--|
| <ol style="list-style-type: none"> 1. Incubating BIs. BIs are depicted as type 'unknown' until the BI Type and Control status are specified by the user on the BI Details page. Navigate to the BI Details page for a specific BI by clicking on the BI image when either the remaining minutes of incubation or result is displayed. 2. Click the virtual button to stop Display flashing and silence the audible alarm (if enabled). A diagonal line through the speaker image indicates the audible alarm has been disabled. 3. Click to minimize the application. 4. Click to close the application. | <ol style="list-style-type: none"> 5. Click to navigate to the History page. 6. Click to navigate to the Settings page. 7. Auto-reader model # 8. Auto-reader firmware version 9. Auto-reader serial number 10. Any BI results, remaining incubation time, or caution/error codes appearing on the Auto-reader Display are shown in this row. Click on caution/error codes for an explanation of the problem and solution. |
|--|--|

HOME PAGE – Firmware Update

- To check for firmware updates, connect the 3M™ Attest™ Auto-reader 490M to your PC when all incubation wells are empty. Open your PC's System Tray, right click on the 3M™ Attest™ Connect icon , and click on **About**. Click on the **Check for Updates** button. If 3M has released a firmware update, an **Install Update** button will appear on the Home page. Click the **Install Update** button to initiate the firmware update.

Notes:

- Firmware updates are available only to registered users who have installed 3M™ Attest™ Connect. You will be prompted to install the latest version of 3M™ Attest™ Connect software, if an update is available, before installing the firmware update.
- The 3M™ Attest™ Mini Auto-reader 490M must be empty (i.e., have no incubating BIs) when a firmware update is installed.

HISTORY PAGE

Navigate to the **History** page by clicking the **History** button on the Home page. The History page provides a summary of the data for all incubated BIs. While the 3M™ Attest™ Mini Auto-reader 490M saves only the last 1,250 records, 3M™ Attest™ Connect can save all records. Records are numbered sequentially. Any positive (+) results for BIs which have not been designated a control BI are highlighted in red (e.g., Record 11 in screenshot). The data in the History page can be sorted by clicking on any of the column headers.

For additional analysis, if desired, click the **Export** button to create a .csv file which will be assigned a unique file name and stored in C: > 3MAttestConnect. The exported file includes both the Auto-reader Values and the User Values for each record.

There are two ways to navigate to a specific **BI Details** page from the **History** page:

- Find the desired record, highlight the row, and click **Details**.
- Find the desired record and double click anywhere in the row.

Record	Well	Start Time	Result	BI Type	Control
24	2	08/29/2019 16:22:07	-	1295	
23	3	08/29/2019 14:59:57	-	1492V	
22	2	08/29/2019 10:21:42	-	1295	
21	1	08/29/2019 08:47:04	-	1295	Control
20	4	08/29/2019 08:44:49	+	1492V	Control
19	2	08/28/2019 17:41:26	-	1295	
18	2	08/28/2019 16:05:33	-	1295	
17	2	08/28/2019 14:35:05	+	1295	
16	2	08/28/2019 13:27:58	-	1295	
15	3	08/28/2019 12:27:11	-	1492V	
14	3	08/28/2019 12:06:51	-	1492V	
13	1	08/28/2019 08:54:21	+	1295	Control
12	4	08/28/2019 08:47:25	+	1492V	Control
11	2	08/27/2019 13:39:47	+	1295	
10	3	08/27/2019 13:34:03	-	1492V	
9	3	08/27/2019 09:53:41	-	1492V	
8	1	08/27/2019 07:57:27	+	1295	Control
7	4	08/27/2019 07:51:03	-	1492V	Control
6	3	08/26/2019 12:48:39	-	1492V	
5	3	08/26/2019 09:02:52	+	1492V	

BI DETAILS PAGE

This page is used to document BI type, BI lot number, control status, and associate a processed BI with a specific sterilization cycle. Only one **BI Details** Page can be open at a time.

The top five fields (Auto-reader Values) are populated automatically by the 3M™ Attest™ Mini Auto-reader 490M and cannot be changed by the user. The remaining fields (User Values) are completed by the end-user.

3M™ Attest™ Connect learns entered values to create a drop-down list of entries specific to your facility. Selections may be eliminated from the dropdown list by right-clicking on the targeted value and responding **Yes** to the Delete message. The data entered in the User Values section of the **BI Details** page may be edited before or after the data is saved.

IMPORTANT: To create a permanent record, click **Print** to generate an individual BI record which can be printed from a networked printer and saved as a paper file or saved as an electronic record (file is saved to C: > 3MAttestConnect).*

* Refer to your facility's record keeping policy and procedure.

The screenshot shows the BI Details page with the following fields:

- Auto-reader Values: Result (-), Start Time (08/28/2019 10:02:57), Well Number (3), 490M Serial Number (567190), Record (5)
- User Values: BI Type (1492V), Control (Control), BI Lot Number (33307G), Sterilizer Number (1), Load Number (2), Implant (checked), Technician (self), Sterilizer Model (Amisco Evolution), Cycle Type/Cycle Parameters (2700, prevac, 4 and 30), Chemical Integrator/Indicator (Pass)
- Buttons: Print, Save, Close

SETTINGS PAGE

Navigate to the Settings page by clicking the Settings button on the Home page. On the **Settings** page the user can:

- Disable or Enable the audible alarm by selecting the desired radio button. The factory setting is alarm enabled. If the audible alarm is disabled, the 3M™ Attest™ Mini Auto-reader 490M display will continue to flash to alert the user to positive (+) BI results and any caution/error codes.
- Synchronize the 3M™ Attest™ Mini Auto-reader 490M time with that of the PC to which it is connected by clicking the **Synchronize Time** button. To automatically synchronize the time each time the unit is connected to a PC, check the **Auto-Sync Time** box.
- Check information (e.g., Serial Number, Heater Block Temperature) about the 3M™ Attest™ Mini Auto-reader 490M.
- Create a file that can be sent to 3M for troubleshooting by clicking the **Service Pack** button. The default storage location for the .spz file is in C: > 3M AttestConnect.

UPGRADING 3M™ ATTEST™ CONNECT SOFTWARE

3M may occasionally provide software updates for the 3M™ Attest™ Connect Auto-reader application. To check for updates, open your PC's System Tray, right click on the 3M™ Attest™ Connect icon , and click on **About**. Click on the **Check for Updates** button to determine whether updates are available. Follow the prompts to download and install the software update. A 3M™ Attest™ Mini Auto-reader 490M does not need to be connected to the PC to download or install 3M™ Attest™ Connect updated software.

TROUBLESHOOTING - 3M™ ATTEST™ CONNECT

Issue	Cause	Solution
BI image on Home page does not match cap and/or process indicator color of incubating BI	User has not specified BI Type and/or user has not designated whether BI is a control BI.	Navigate to BI Details page and specify BI Type from the dropdown list; check the control box for control BIs.
Home page is not displayed on PC screen	PC doesn't detect connected 3M™ Attest™ Auto-reader 490M	Unplug and then reconnect USB cable to PC.
Home page not displayed on PC screen	3M™ Attest™ Connect has been minimized.	Click on the  icon in the PC task bar.
Home page is not displayed on PC screen	3M™ Attest™ Connect has been closed.	Open PC System Tray, right click on the 3M™ Attest™ Connect icon  , click on applicable 3M™ Attest™ Auto-reader 490M serial number.
Home page is not displayed on PC screen and  icon doesn't appear in PC System Tray	Application may have crashed.	On PC, go to Search Windows , type 3M, and click on  3M™ Attest™ Connect Or Go to Windows Start menu, 3M folder, click on 3M™ Attest™ Connect.
On History page, BI Start Times read the year 2000	3M™ Attest™ Mini Auto-reader 490M real time clock's back-up battery is not operating correctly.	Contact Customer Support. The real time clock captures BI incubation start time but does not control incubation duration. The accuracy of BI results is not affected by failure of the real time clock's back-up battery.
Pop-up appears when user clicks Install Update button on 3M™ Attest™ Connect Home page.	3M™ Attest™ Connect software must be current before 3M™ Attest™ Auto-reader 490M firmware can be upgraded.	Follow instructions to update 3M™ Attest™ Connect software. Then click Install Update button on 3M™ Attest™ Connect Home page and proceed with firmware update.

CUSTOMER SUPPORT

In the USA, call the 3M Health Care Help Line at 1-800-228-3957.

Outside of the USA, contact your local 3M Subsidiary.

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© 2020, 3M. All rights reserved.
3M Deutschland GmbH