
How to use

3M™ Curoso™ Tips Disinfecting Cap Strip for Male Luers

Disinfects in 1 minute. Protects male luers for up to 7 days.

Peel Curoso Tips Disinfecting Cap from foil strip.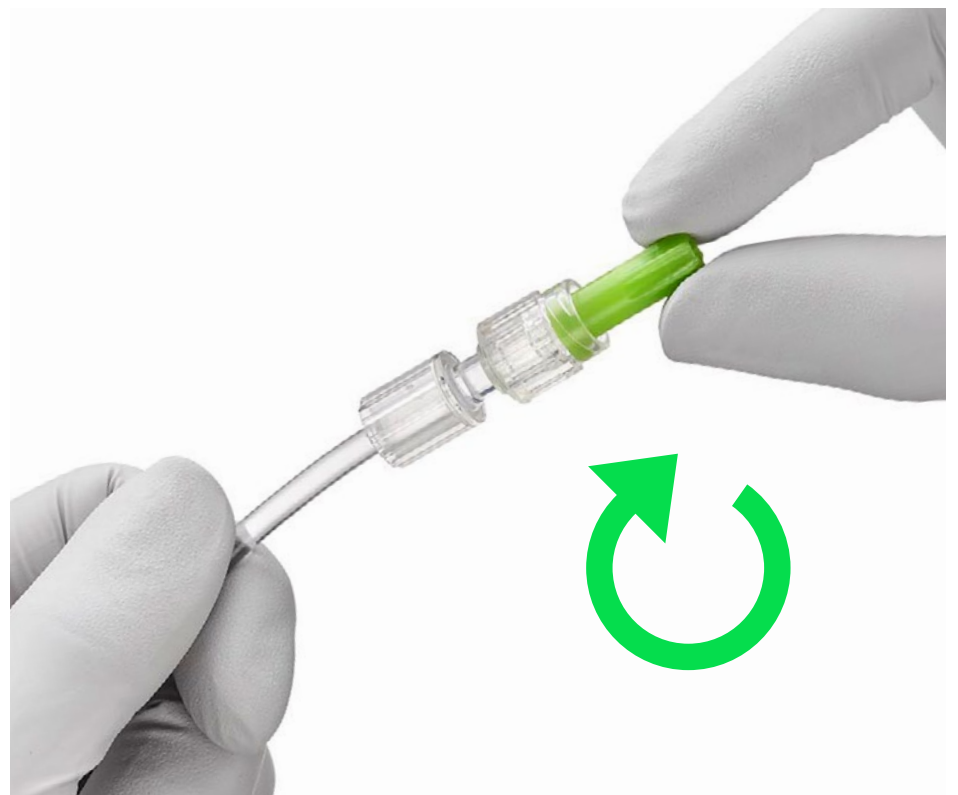

Push and twist the Curoso Tips Disinfecting Cap clockwise onto the end of the male luer.

Removal: The male luer connector can be accessed immediately after removing the Curoso Tips Disinfecting Cap that has been in place for a minimum of 1 minute.

- If male luer connection on I.V. line is heavily soiled, clean with 70% isopropyl alcohol swab before using Curoso Tips Disinfecting Cap
- Do not use if foil is broken, torn, punctured, or if Curoso Tips Disinfecting Cap appears dry inside
- Curoso Tips Disinfecting Cap must remain on male luer connection for a minimum of one (1) minute and may remain on for seven (7) days if not removed
- Always place a new Curoso Tips Disinfecting Cap on male luer connection after each use
- Dispose of the Curoso Tips Disinfecting Cap after every use; do not reuse. Reuse may result in cross-contamination and does not provide efficacy due to lack of alcohol present

Caution: Potential choking hazard

 Proven protection that's easy to see

STERILE | R

Available in Canada from:

3M Healthcare Canada Company Limited
75 Courtneypark Drive W., Unit #4
Mississauga, ON L5W 0E3
Canada

Phone 1-800-364-3577
Web solventum.com/en-ca

Active ingredient: 70% isopropyl alcohol (IPA). Single Use Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.

© Solventum 2026. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M is a trademark of 3M. All other trademarks are owned by their respective owners.

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