

**3M™ Scotchbond™**  
**Universal Plus**  
Adhesive

More than  
just a strong  
bond.



Request a demo

# The benchmark adhesive you trust ..

3M™ Scotchbond™ Universal Plus Adhesive takes universal adhesives to the next level. It keeps all the benefits of the original 3M™ Scotchbond™ Universal Adhesive while adding even greater control and predictability – because a good adhesive should provide more than just a strong bond.

**5** steps  
forward from  
the original


- Added radiopacity
- Bonds and seals caries-affected dentine
- Advanced bonding to dental and restorative substrates, including glass ceramics
- Full dual- and self-cure compatibility without additional dual-cure activator bottle
- Free of BPA derivatives like BisGMA



Plus, the original Scotchbond Universal Adhesive's popular features:

## Universally suitable

- ✓ Direct
- ✓ Indirect
- ✓ All etching techniques

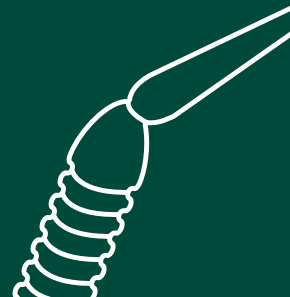


# . now setting a new standard

## Minimally Invasive Dentistry



### Added radiopacity

Radiolucency under an existing restoration could lead to misdiagnosis and invasive overtreatment. 3M™ Scotchbond™ Universal Plus Adhesive is **the first radiopaque universal adhesive** with dentine-like radiopacity, helping to overcome this issue.

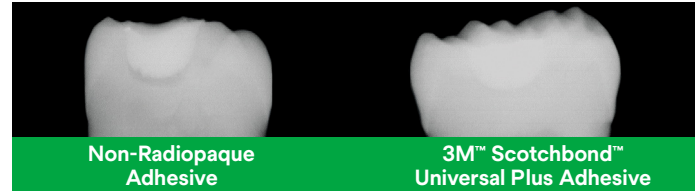

**Intentional adhesive pooling under Class I restorations** – X-rays of extracted human molars, filling material 3M™ Filtek™ One Bulk Fill Restorative. K. Dede et al., J Dent Res 99 (Spec Iss A): 192 IADR, 2020 – 3M internal data, H. Loll et al., J Dent Res 99 (Spec Iss A): No. 757, 2020.

“These doubtful X-rays happen every day; most often critical step in bitewing radiograph, class II in proximal box.”

– Prof. Dr. Angelo Putignano, University of Marche-Ancona, Italy


“Radiolucent adhesive pooling underneath a restoration doubles the risk of restoration replacement.”

– Dr. John O. Burgess, University of Alabama, USA


### Bonds and seals caries-affected dentine to help preserve tooth structure

Scotchbond Universal Plus Adhesive gives you **confidence when performing minimally invasive preparations** because it has the same high bond strength to caries-affected dentine as to sound dentine.

In addition to the excellent quality of the restoration margins, it seals caries-affected dentine by forming a well-defined, void-free hybrid layer – because “the seal is the deal” to prevent secondary decay.

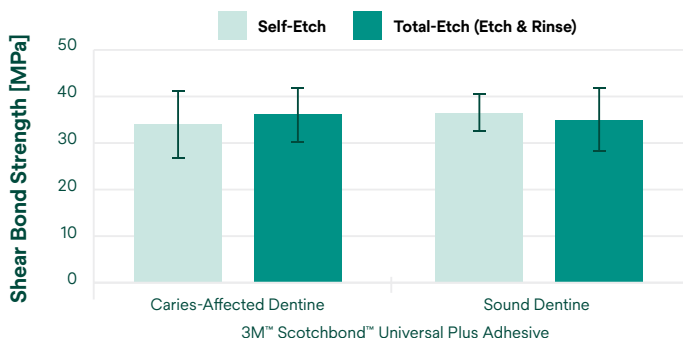

**Shear bond strength on caries-affected and sound dentine** – C. Thalacker, et al., Interaction of an Experimental Universal Adhesive with Caries-Affected Dentin, J Dent Res 99 (Spec Iss A): 191 IADR, 2020.

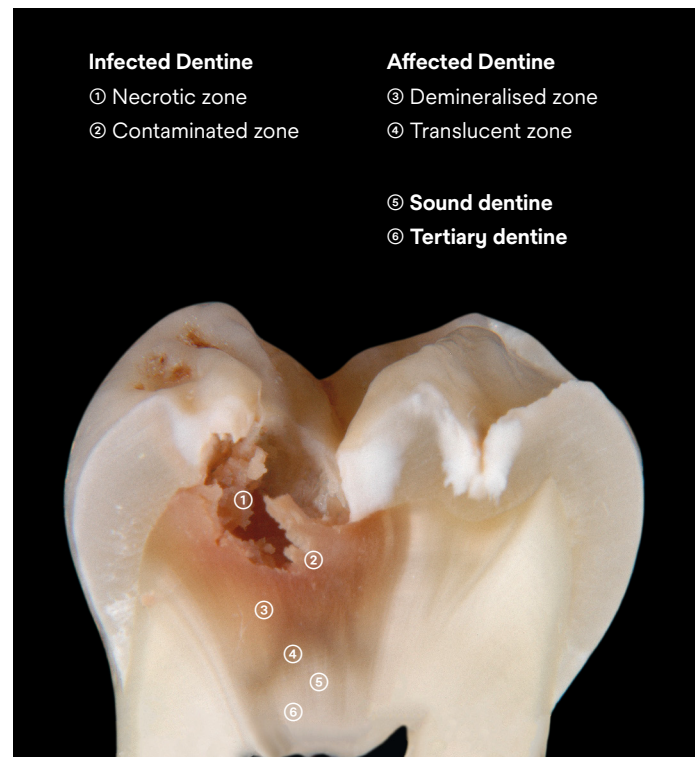

**Caries zones** – Image courtesy of Prof. Leandro Hilgert and Prof. Soraya Leal, University of Brasilia (UnB), Brazil.

### Bonds all restorative materials

without the need for separate primers

### Virtually no post-operative sensitivity


# 99%

of dentists reported that 3M™ Scotchbond™ Universal Plus Adhesive shows virtually no post-operative sensitivity\*

\* Field evaluation with 309 dentists and over 20,000 applications.

# Advanced bonding to glass ceramic

Scotchbond Universal Plus Adhesive reaches the gold standard level of traditional, separate silanes.

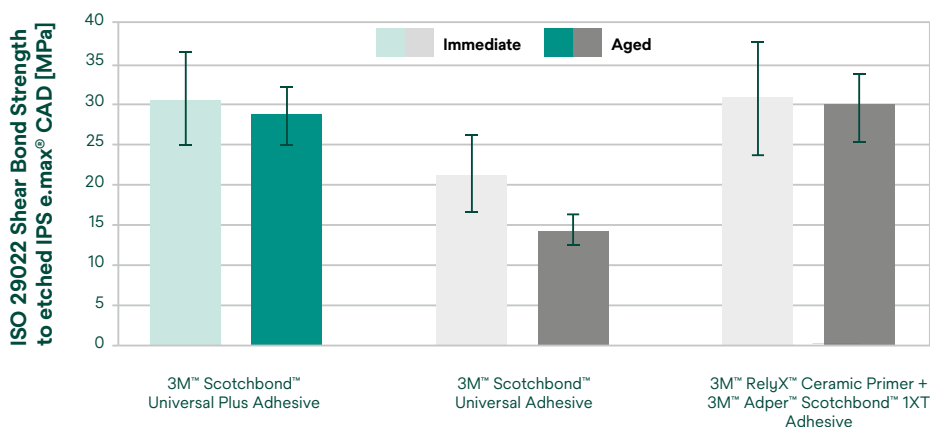

Shear bond strength on glass ceramic – Prof. Dr. Bart van Meerbeek et al., KU Leuven, Belgium, Report to 3M, 2020

## Report conclusion:

3M™ Scotchbond™ Universal Plus Adhesive performed better than 3M™ Scotchbond™ Universal Adhesive for glass ceramic (IPS e.max® CAD, Ivoclar Vivadent) bonding. After the glass ceramic was etched with HF acid, a favorable 'immediate' and 'aged' shear bond strength were recorded.

## Fully aligned system with 3M™ RelyX™ Universal Resin Cement

Excellent bond strength for virtually all dual-cure resin cement indications.† Simplified workflows – 3M Scotchbond Universal Adhesive is cured when in contact with RelyX Universal Resin Cement.



## Ordering Information

### Item no. Product information

#### 3M™ Scotchbond™ Universal Plus Adhesive – Vial

41293	<b>Intro Kit Vial</b> 1 Vial 5ml, 50 Disposable Brushes, 1 Etchant Syringe 3ml, 25 Dispensing Tips
41294	<b>Refill Vial – 1 x 5ml</b>
41295	<b>Refill Vial – 3 x 5ml</b>
41263	<b>3M™ Scotchbond™ Universal Etchant Etching Gel</b> 2 Syringes x 3ml, 50 Dispensing Tips

#### 3M™ Scotchbond™ Universal Plus Adhesive – Unit Dose

41297	<b>Intro Kit Unit Dose</b> 50 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips
41298	<b>Unit Dose 100</b> 100 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips
41299	<b>Unit Dose 200</b> 200 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips
41304	<b>Unit Dose 400</b> 400 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips



**Vial**  
with smooth and even surface  
– easy to wipe off excess



**Unit Dose**  
for efficient hygiene management  
–reduces the risk of cross contamination

[go.solventum.com/sbup-demo-anz](https://go.solventum.com/sbup-demo-anz)



Solventum Dental Solutions

KCI Medical Australia Pty Ltd | Level 3, Building A, 1 Rivett Road, North Ryde, NSW 2113

KCI New Zealand Unlimited | Suite 1701, Level 17, PwC Tower, 15 Customs Street West, Auckland 1010

†Resin Cement Bond Strength to Multiple Substrates. Dental Advisor, M. Cowen, J.M. Powers, 2024. ©Solventum 2025. Solventum and the S logo are trademarks of Solventum or its affiliates. Other trademarks are the property of their respective owners. PSHCBESP0004844.



**Request a demo**