



# Filtek™

Bulk Fill  
Flowable Restorative

## The bulk fill flowable that's easier to inject.

3M™ Filtek™ Bulk Fill Flowable Restorative is available in an ergonomic syringe designed for comfort and easier injecting.<sup>1</sup> This gives you more control and visibility for precise placement in deep restorations.

### Reduce bubbles and waste

Virtually no bubbles or material “run-on” during dispensing gives you better control.\*

### No more guessing

The green syringe barrel indicates the remaining material volume. When your syringe is empty, only the white plunger shows.



### Bendable cannula

Easy-to-bend without kinking for better access.\*

### Easy to hold and inject

Triangle finger plate and plunger make it quick and easy to your grip.\*

## Key benefits

- Virtually no bubbles or material “run-on”\*
- Easy-to-bend cannula for improved access\*
- Ergonomic design is easy to hold and inject\*
- Injectable, one-step placement up to 4 mm\*
- Excellent adaptation and low shrinkage stress\*

## Can be warmed for injection moulding

Warm up to 70°C for up to one hour, up to 25 times.\*



## Posterior restoration made simple



Prepare the tooth and apply an adhesive e.g. 3M™ Scotchbond™ Universal Adhesive.



Dispense up to 4 mm of 3M™ Filtek™ Bulk Fill Flowable Restorative into the prepared cavity and cure, e.g. with 3M™ Elipar™ DeepCure-S LED Curing Light.



Place a 2 mm (minimum) capping layer of an aesthetic composite, such as a 3M™ Filtek™ Universal Restorative, and cure.

Clinical photos courtesy of Dr. Reich, Seefeld, Germany

## Indications

(Incremental up to 4 mm)



Base  
(Class I – II)



Liner  
(Class I – II)



Minimally Invasive  
Dentistry (MID)<sup>1</sup>



Class III



Class V



Repair of Indirect  
(small defects)



Repair<sup>2</sup>



Undercut  
Block-out



Pit & Fissure  
Sealant



Core  
Build-up<sup>3</sup>

Refer to Instructions  
for Use for full list of  
indications.

## Ordering information



### 3M™ Filtek™ Bulk Fill Flowable Restorative Refills

Syringe refills include: 2 – 2g syringes; 20 dispensing tips.

SHADE	ITEM NO.
A1	4862A1
A2	4862A2
A3	4862A3
Universal	4862U

ITEM NO.	PRODUCT INFO.
3740T	3M™ Filtek™ Bulk Fill Flowable Restorative Dispensing Tips, 19-gauge, Green (20)



### References

1. Including small, non stress-bearing occlusal restorations.
2. Repair of acrylic temporary and composite resin.
3. Where at least half the coronal tooth structure is remaining to provide structural support for the crown.

\*3M Internal data



3M Oral Care  
3M United Kingdom PLC  
Charnwood Campus  
10 Bakewell Road  
Loughborough  
Leicestershire, LE11 5RB  
+0800 626 578

3M Ireland Ltd  
The Iveagh Building  
Carrickmines Park  
Carrickmines  
Dublin 18  
Ireland  
+353 1 280 3555

3M.co.uk/Dental

3M Elipar, Scotchbond and Filtek are trademarks of the 3M company. © 3M 2020. All rights reserved. 13432884-200210-G0111A