

3M™ Steri-Drape™ 1115 Tube Organizer



Organize

Your Sterile Field

The innovative 3M Steri-Drape 1115 Tube Organizer is a simple way to organize tubes and cords within a sterile field.

- Securely positions and organizes multiple tubes and cords of different sizes
- Conveniently adheres to tables, stands, drapes and other OR equipment
- Individually wrapped and sterilized in peel-open envelopes to maintain the sterile environment

Lead
The Way


3M

3M™ Steri-Drape™ 1115 Tube Organizer

Puts tubes and cords in their place so you can focus on the procedure.



Reclosable adhesive lets you easily open and close tab to reposition, add or replace tubes and cords.

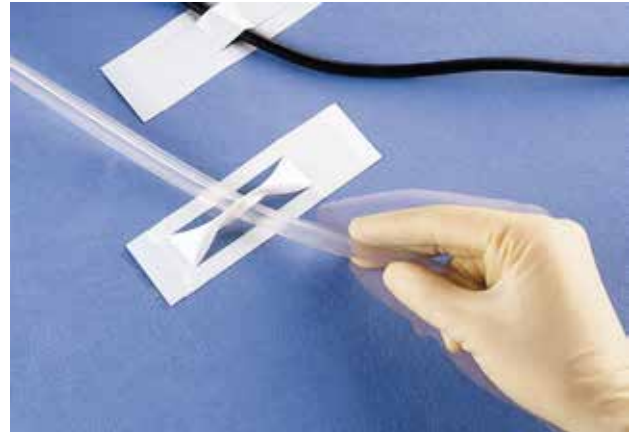

Tubes and cords can move freely between the adhered ends if you pinch together the middle portion of the adhesive tab.

Application Instructions



1. Peel back white tab to remove adhesive liner.



2. Apply organizer(s) to desired area. Peel back blue tab to open organizer.



3. Place tube(s) and/or cord(s) in desired position. To securely hold tubes/cords in position, return blue tab to original position, ensuring adhesive holds in place. If tubes/cords need to move freely during the procedure, pinch together the middle portion of the adhesive tab (see photo, top right) and return blue tab to original position.



4. If removal of organizer is necessary, lift from closed end of tab.

Ordering Information

3M™ Steri-Drape™ 1115 Tube Organizer

Cat No.	Product Description	Packaging	Minimum Order	Pricing Unit
1115	5 1/4" x 1 1/2" (133cm x 38cm)	1 Each 100 Each/Box 4 Box/Case	Case	Case

For more information about 3M™ Surgical Drapes or other 3M Health Care products:

- » Contact your local 3M representative
- » Call the 3M Health Care Customer Helpline at 800-228-3957
- » Visit www.3M.com/infectionprevention



Infection Prevention Division 3M Health Care

2510 Conway Avenue
St. Paul, MN 55144-1000
U.S.A.
1-800-228-3957
www.3M.com/infectionprevention

3M and Steri-Drape are trademarks of 3M.
©3M 2014. All rights reserved.
70-2009-9532-5