

New and advanced features include:

• Easy custom configuration buttons on top of the unit

- Remote viewing of BI results from your Smart Phone or PC
- · Saving or printing incubation records from web app
- Ability to connect to select instrument-tracking software systems
- Backlit LCD panel for fast, easy reading of data, including at-a-glance minutes remaining in incubation



The 3M[™] Attest[™] Auto-reader 390 and 390G

CUSTOM CONFIGURATION

- Press once to see time, date, serial number, firmware version and Auto-reader configuration
- Press again to view IP address and date and time that power was supplied to the unit
- Use up and down arrow buttons with on-screen guidance to custom configure settings (or use the internet and your PC)

TEMPERATURE CHECK

 Press once to see current heater block temperature

VISIBLE INDICATOR-

 A plus (+) symbol appears as soon as a positive result is detected

MINUTES LEFT IN — INCUBATION CYCLE

Use the products below with the 3M[™] Attest[™] Auto-reader 390:



1291 & 1292 3M™ Attest™ Rapid Readout Steam Biological Indicators



1296 & 1296F 3M[™] Attest[™] Rapid Readout Steam Biological Indicator Test Packs



41382 & 41382F 3M™ Attest™ Rapid 5 Steam-Plus Test Packs



- LCD display flashes when a positive result is detected
- Alarm also sounds if a 3M[™] Attest[™] Rapid Readout Biological Indicator is removed from the reader before final results have been detected

Use the products below with the 3M[™] Attest[™] Auto-reader 390G:



1294 3M™ Attest™ Rapid Readout E0 **Biological Indicator**



1298 & 1298F 3M™ Attest™ Rapid Readout EO **Biological Test Packs**

Ordering Information

Catalog No.	Description	Packaging	Shipping
Steam Sterilization			
390	3M [™] Attest [™] Auto-reader 390 for Steam Sterilization Use with 3M [™] Attest [™] Rapid Readout Biological Indicators for Steam 1291 and 1292, 60°C (140°F)	1 unit/box	1 box/case
1291	3M [™] Attest [™] Rapid Readout Biological Indicator For 132°C (270°F) Gravity Displacement Steam Sterilization (Blue Cap)	50 each/box	4 boxes/case
1292	3M [™] Attest [™] Rapid Readout Biological Indicator For 121°C (250°F) Gravity and 132°C (270°F) Vacuum-Assisted Steam Sterilization (Brown Cap)	50 each/box	4 boxes/case
1296 1296F	3M [™] Attest [™] Rapid Readout Biological Test Packs For 121°C (250°F) Gravity and 132°C (270°F) Vacuum-Assisted Steam Sterilization	25 packs/box 25 controls/box 25 packs/box 5 controls/box	1 box/case 1 box/case
41382 41382F	3M [™] Attest [™] Rapid 5 Steam-Plus Test Packs Monitors 132°C (270°F), ≥4-minute vacuum-assisted and 121°C (250°F), ≥30-minute gravity displacement steam sterilization processes. Each pack contains a 3M [™] Attest [™] Rapid Readout Biological Indicator 1292 and a 3M [™] Comply [™] (SteriGage [™]) Steam Chemical Integrator.	16 each/box 16 controls/box 16 each/box 4 controls/box	4 boxes/case 4 boxes/case
Ethylene Oxide (EO) Sterilization			
390G	3M [™] Attest [™] Auto-reader 390G for Ethylene Oxide (EO) Sterilization Use with 3M [™] Attest [™] Rapid Readout BI 1294, 37°C (98.6°F)	1 unit/box	1 box/case
1294	3M [™] Attest [™] Rapid Readout Biological Indicator For Ethylene Oxide (E0) Sterilization (Green Cap)	50 each/box	4 boxes/case
1298	3M [™] Attest [™] Rapid Readout Biological Test Packs For Ethylene Oxide (EO) Sterilization	25 packs/box 25 controls/box	2 boxes/case
1298F 3IVI Health Care Infection Prevention	Division	25 packs/box 5 controls/box	2 boxes/case
2510 Conway Avenue St. Paul, MN 55144-100 USA 1-800-228-3957			

1-800-228-3957 www.3M.com/infectionprevention