

# 3M™ Cavilon™ No Sting Barrier Film Application Guide for I.V. Site Protection




3M™ Cavilon™ No Sting Barrier Film 1 mL wand applicator is recommended for skin protection around intravenous catheter sites.

**NOTE!** The 28 mL spray bottle should not be used for IV site protection.

## Application at time of catheter insertion:

1. **Prepare for and follow catheter insertion** procedure per facility protocol.
2. **Apply antiseptic prep** and allow to dry completely. ⌚
3. **Insert catheter.**
4. **Apply barrier film:** Avoiding the area immediately surrounding insertion site (approximately 2cm), apply an even coating of Cavilon No Sting Barrier Film to the area that will be covered by the adhesive dressing/device. If using Tegaderm™ CHG Chlorhexidine Gluconate I.V. Securement Dressing do not apply the film to skin where the CHG gel pad is placed.
5. **Allow Cavilon No Sting Barrier Film to dry completely** (approximately 30 seconds). ⌚
6. **Apply dressing:** Tegaderm™ I.V. Advanced Securement Dressing or Tegaderm™ CHG Chlorhexidine Gluconate I.V. Securement Dressing or 3M™ PICC/CVC Securement System per protocol.

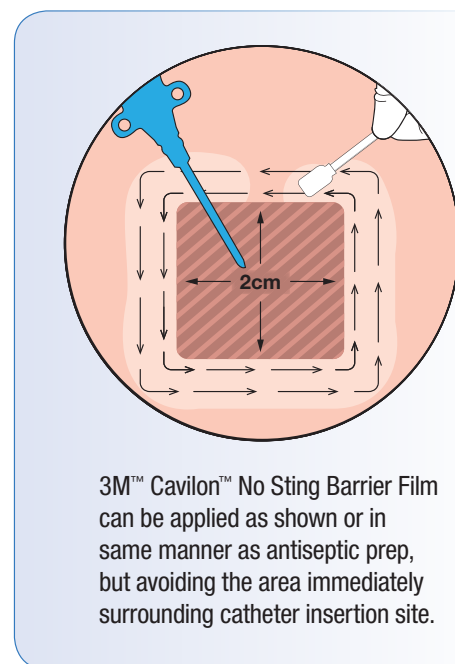

3M™ Cavilon™ No Sting Barrier Film can be applied as shown or in same manner as antiseptic prep, but avoiding the area immediately surrounding catheter insertion site.

## Application at dressing change:

1. **Prepare for and follow dressing change procedure** per facility protocol.
2. **Remove securement dressing/device.**  
**NOTE!** Cavilon No Sting Barrier Film forms a protective interface between skin and the adhesive. It is removed with the adhesive product and must be reapplied with each dressing change.
3. **Apply antiseptic prep** and allow to dry completely. ⌚
4. **Apply barrier film:** Avoiding the area immediately surrounding insertion site (approximately 2cm), apply an even coating of Cavilon No Sting Barrier Film to the area that will be covered by the adhesive dressing/device. If using Tegaderm™ CHG Chlorhexidine Gluconate I.V. Securement Dressing do not apply the film to skin where the CHG gel pad is placed.
5. **Allow Cavilon No Sting Barrier Film to dry completely** (approximately 30 seconds). ⌚
6. **Apply dressing:** Tegaderm™ I.V. Advanced Securement Dressing or Tegaderm™ CHG Chlorhexidine Gluconate I.V. Securement Dressing or 3M™ PICC/CVC Securement System per protocol.



**Critical & Chronic Care Solutions Division**  
**3M Health Care**  
2510 Conway Avenue  
St. Paul, MN 55144  
USA  
1-800-228-3957  
www.3M.com/Cavilon

**3M Canada**  
P.O. Box 5757  
London, Ontario N6A 4T1  
Canada  
1-800-364-3577  
www.3m.ca/skinwound

3M and Cavilon are trademarks of 3M. Used under license in Canada.  
Please recycle. Printed in USA. © 3M 2014. All rights reserved.  
70-2011-5546-5

⌚ = allow dry time