



Curoso™

Disinfecting Port Protectors

HOW TO USE


Curoso™ for Tego®

Disinfecting Cap for Tego® Hemodialysis Connectors

Disinfects in 1 minute — protects ports for up to 7 days.



Peel
the foil from the 3M™ Curoso™ Disinfecting Cap for Tego® Hemodialysis Connector.

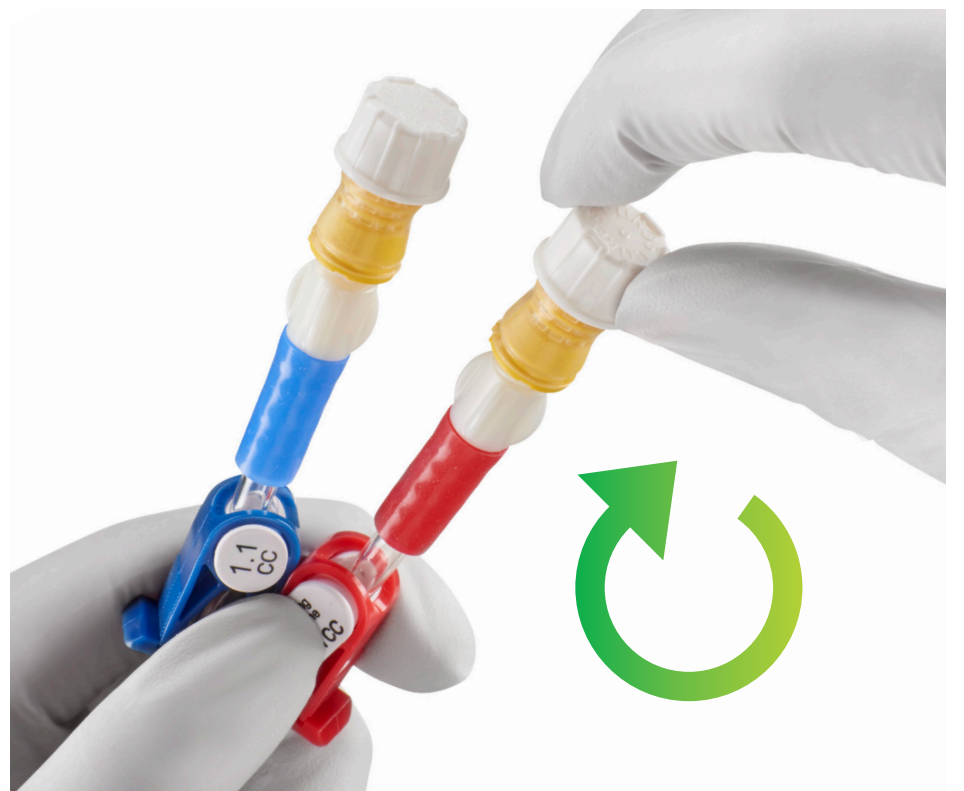

Push & Twist
the Curoso disinfecting cap for Tego® hemodialysis connector clockwise onto the end of the hemodialysis connector.

- If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab prior to using cap.
- Do not use cap if seal is broken, torn, punctured, or sponge appears dry.
- Cap must remain on needleless connector for minimum of one (1) minute and may remain on for seven (7) days if not removed.
- The needleless connector can be accessed immediately after removing the Curoso disinfecting cap for Tego® hemodialysis connector that has been in place for a minimum of one (1) minute.
- Always place a new Curoso disinfecting cap for Tego® hemodialysis connector on hemodialysis connector after each use.
- Dispose of the Curoso disinfecting cap for Tego® hemodialysis connector after every use.
- Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses.

WARNING: To avoid potential for injury, use only on Tego® Hemodialysis Connectors.

CAUTION: Potential choking hazard.



Critical & Chronic Care Solutions Division
3M Health Care
2510 Conway Avenue
St. Paul, MN 55144 USA

1 800 228-3957
www.3M.com/C3SD

Please recycle this poster. Printed in U.S.A.
© 3M 2016. All rights reserved.
3M and Curoso are trademarks of 3M.
Tego is a registered trademark of ICU Medical.
70-2011-5799-0

STERILE R

Active Ingredient: 70% isopropyl alcohol (IPA).
Single Use Only. Rx Only. This device is not
made with natural rubber latex or plasticiser
Diethylhexylphthalate (DEHP). Refer to current
device labeling for indications for use, warnings
and product details.