

3M™ Comply™

Hydrogen Peroxide Indicator Tape 1228*

*For use in STERRAD® 100, STERRAD® 100S, STERRAD® NX®, STERRAD® 100NX® and AMSCO® V-PRO™ 1, AMSCO® V-PRO™ 1 Plus, and AMSCO® V-PRO™ maX Low Temperature Sterilization System.

Product Description

3M™ Comply™ Hydrogen Peroxide Indicator Tape 1228 consists of a non-cellulosic plastic backing material with a pressure-sensitive adhesive on one side and chemical indicator stripes on the other side. The chemical indicator stripes turn from blue to pink after exposure to vaporized hydrogen peroxide.

Indications for Use

Use the 3M™ Comply™ Hydrogen Peroxide Indicator Tape 1228 to secure packs and as an external pack process indicator to differentiate unprocessed items from items processed in the STERRAD® 100, 100S, NX® (Standard and Advanced cycles) and 100NX® (Standard, Flex, Express and Duo cycles) sterilization processes and AMSCO® V-PRO™ 1 (Lumen cycle), AMSCO® V-PRO™ 1 Plus (Lumen and Non Lumen cycles), and AMSCO® V-PRO™ maX Low Temperature Sterilization System (Lumen, Non Lumen, and Flexible cycles) sterilizers. The tape is suitable for use on non-woven disposable wraps and peel pouches. The chemical indicator stripes turn from blue to pink after exposure to vaporized hydrogen peroxide.

Contraindications

None.

Precautions

1. DO NOT use the 3M™ Comply™ Hydrogen Peroxide Indicator Tape 1228 in sterilization processes which it is not designed to monitor, such as:
 - Steam sterilization cycles,
 - Dry heat sterilization cycles,
 - Ethylene oxide sterilization cycles, or
 - Other chemical vapor or low temperature sterilization processes.
2. To prevent color change of the indicator ink before use, do not store near sterilants or other chemicals.

Instructions for Use


1. Open the plastic package wrap that contains the roll of tape and install in a tape dispenser.
2. Unwind a section of indicator tape long enough to secure the pack.
3. Apply the tape using finger pressure along the entire length of the tape to secure the pack. Apply tape to the pack so that half the length of tape is on each side of the fold to be secured.
4. Repeat steps 2 and 3 for additional lengths of tape that are applied to the pack.
5. After sterilization, verify that the indicator stripes have turned pink. If the indicator stripes have not turned pink, the pack should not be used.
6. When the pack is ready for use, open the pack by breaking the tape at the point of the fold flap. Do not remove tape from wraps in the operating room to guard against linting.


Storage


- Best stored at room temperature (15 -25°C [59 - 77°F]) and relative humidity (RH) <50%.
- After use, the indicator will not change visually within 12 months when stored at above conditions.

Explanation of Symbols

 Catalogue Number

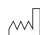
 Caution, see instructions for use


 Do not reuse

 Use by date

 Batch code

 Manufacturer

 Date of manufacture

 Product is designed for use with vaporized hydrogen peroxide sterilization processes

Made in U.S.A. by

 **3M Health Care**

2510 Conway Ave.

St. Paul, MN 55144

1-800-228-3957

3M.com/infectionprevention

3M and Comply are trademarks of 3M.

Used under license in Canada.

© 2017, 3M. All rights reserved.

STERRAD, 100NX and NX are registered trademarks of Advanced Sterilization Products, Division of Ethicon Inc., a Johnson & Johnson company.

AMSCO and V-PRO are registered trademarks of Steris Corporation.

Issue Date: 2017-08

34-8721-6397-6