

# The bulk fill flowable that's easier to inject.

3M<sup>™</sup> Filtek<sup>™</sup> Bulk Fill Flowable Restorative is available in a new ergonomic syringe designed for improved comfort and easier injecting. This gives you more control and visibility for precise placement in deep restorations.



Virtually no bubbles or material "run-on" during dispensing gives you better control.



The green syringe barrel indicates the remaining material volume. When your syringe is empty, only the white plunger shows.

3M | Filtek<sup>-</sup> Bulk Fill Flowable Restorative



Bendable cannula Easy-to-bend without Easy to hold and inject

Triangle finger plate and plunger make it quick and easy to personalize your grip.

### **Key benefits**

- Virtually no bubbles or material "run-on"
- Easy-to-bend cannula for improved access
- Ergonomic design is easy to hold and inject
- Injectable, one-step placement up to 4 mm
- Excellent adaptation and low shrinkage stress

## Can be warmed for injection molding

Warm up to 70°C (158°F) for up to one hour, up to 25 times.



#### Posterior restoration made simple





Prepare the tooth and apply an adhesive e.g. 3M<sup>™</sup> Scotchbond<sup>™</sup> Universal Adhesive.

Dispense up to 4 mm of 3M<sup>™</sup> Filtek<sup>™</sup> Bulk Fill Flowable Restorative into the prepared cavity and cure, e.g. with 3M<sup>™</sup> Elipar<sup>™</sup> DeepCure-S LED Curing Light.

Place a 2 mm (minimum) capping layer of an aesthetic composite, such as a 3M<sup>™</sup> Filtek<sup>™</sup> Universal Restorative, and cure.

Clinical photos courtesy of Dr. Reich, Seefeld, Germany

#### Indications (Incremental up to 4 mm) **Minimally Invasive** Base Liner (Class I – II) (Class I – II) Dentistry (MID)<sup>1</sup> Class V Class III **Repair of Indirect** (small defects) Undercut Repair<sup>2</sup> Pit & Fissure **Block-out** Sealant **Refer to Instructions** for Use for full list of indications. Core

#### **Ordering information**

Build-up<sup>3</sup>



3M<sup>™</sup> Filtek<sup>™</sup> Bulk Fill Flowable Restorative Refills Syringe refills include: 2 – 2g syringes; 20 dispensing tips.

SHADE	ITEM NO.
A1	4862A1
A2	4862A2
A3	4862A3
Universal	4862U

ITEM NO.	PRODUCT INFO.	1
3740Т	3M <sup>™</sup> Filtek <sup>™</sup> Bulk Fill Flowable Restorative Dispensing Tips, 19-gauge, Green (20)	

**3**M

3M Oral Care 2510 Conway Avenue St. Paul, MN 55144-1000 USA

Phone 1-800-634-2249 Web 3M.com/dental 3M Canada Post Office Box 5757 London, Ontario N6A 4T1 Canada

Phone 1-888-363-3685

1. Including small, non stress-bearing occlusal restorations.

- 2. Repair of acrylic temporary and composite resin.
- 3. Where at least half the coronal tooth structure is remaining to provide structural support for the crown.

3M, Elipar, Filtek and Scotchbond are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Please recycle. Printed in U.S.A. © 3M 2019. All rights reserved. 70-2013-7232-6