



**3M** Science.  
Applied to Life.™

**3M™ Vanish™  
5% Sodium Fluoride  
White Varnish**

**Extended fluoride  
release varnish ...  
because staying in  
contact matters.**



Most patients don't know the benefit of keeping a fluoride varnish in contact with their teeth longer.

# Studies show the longer a fluoride varnish stays in contact with the teeth, the more effective it is.\*

That's why 3M formulated Vanish White Varnish to stay on the teeth longer for more effective protection.




Durable, long-lasting coating



Releases **more fluoride** after 4 hours than other leading varnishes, continuously releasing up to 24 hours

Extended-release formula allows fluoride, calcium and phosphate to be released over a longer period, allowing absorption into the tooth.

Contains 22,600 ppm fluoride and tri-calcium phosphate. Minerals found in saliva for building strong teeth.

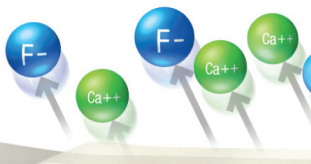

Vanish White Varnish deposits strong, acid-resistant mineral on tooth surfaces—including dentin tubules—to relieve hypersensitivity

## 3M™ Vanish™ 5% Sodium Fluoride White Varnish



The #1-selling varnish in the U.S.

High fluoride uptake, even after acid attack.

Vanish White Varnish



Test results show the high fluoride uptake during a 24-hour treatment of Vanish White Varnish is virtually unaffected by acid challenge.

Source: 3M internal data

\*Source: Karlinsey, Robert L. (2016). Fluoride Varnishes: Why They Work & What to Look For. *EC Dental Science*, 5.6, 1220–1223.



# Advanced varnish with proven protection is fast and easy to apply.

**Simply swipe 3M™ Vanish™ 5% Sodium Fluoride White Varnish across tooth surfaces and let the formula do the rest.**

- No need to dry the teeth—sets in the presence of saliva
- No rinsing—the patient may eat and drink immediately after the application
- Saliva-tolerant—spreads to help coat even more tooth surfaces, releasing fluoride, calcium and phosphate for up to 24 hours



Dispense Vanish White Varnish onto a mixing surface. For easy application, squeeze it onto the gloved hand like a painter's palette, so you can efficiently deliver varnish to the patient.

## Summary of advantages

- Extended fluoride release—22,600 ppm sodium fluoride
- Contains calcium, phosphate and xylitol
- Easy to apply, no drying needed
- Can be applied to tooth surfaces where plaque is present
- Fast, horizontal sweep application
- Unique applicator for back-of-glove dispensing
- Does not change the appearance of ceramic or metal orthodontic brackets
- Relieves hypersensitivity
- Over 10 years of clinical success



Apply Vanish White Varnish evenly in a thin layer over treatment area(s) with sweeping, horizontal brush strokes. No suction required.

# It's the varnish that won't leave their teeth when they leave your chair.

**3M™ Vanish™ 5% Sodium Fluoride White Varnish** is virtually invisible on teeth. It leaves a durable, thin coating that adheres to the teeth until it's brushed off, creating long-lasting, acid-resistant protection to relieve hypersensitivity.

**Proven, advanced protection you can't see.**



Immediately after Vanish White Varnish was applied.



## Ordering Information

Item #	Product Information
	<b>3M™ Vanish™ 5% Sodium Fluoride White Varnish—50 Pack</b> Each 50 Pack includes: 50 unit-dose packets (0.5ml each) of one flavor and applicator brushes in individual foil pouches
12149C	<b>Cherry Flavor</b>
12149L	<b>Melon Flavor</b>
12149M	<b>Mint Flavor</b>
	<b>3M™ Vanish™ 5% Sodium Fluoride White Varnish—100 Pack</b> Each 100 Pack includes: 100 unit-dose packets (0.5ml each) of one flavor; 100 applicator brushes
12150C	<b>Cherry Flavor</b>
12150L	<b>Melon Flavor</b>
12150M	<b>Mint Flavor</b>
	<b>3M™ Vanish™ 5% Sodium Fluoride White Varnish—Bulk Pack</b> Each Bulk Pack includes: 10 boxes of one flavor. Each box includes: 100 unit-dose packets (0.5ml each); 100 applicator brushes
12151C	<b>Cherry Flavor</b>
12151L	<b>Melon Flavor</b>
12151M	<b>Mint Flavor</b>
12151CL	<b>3M™ Vanish™ 5% Sodium Fluoride White Varnish Bulk Pack—Cherry &amp; Melon Flavors</b> Includes: 10 boxes (5 Cherry, 5 Melon). Each box includes: 100 unit-dose packets (0.5ml each); 100 applicator brushes
12151X	<b>3M™ Vanish™ 5% Sodium Fluoride White Varnish Bulk Pack—Assorted Flavors</b> Includes: 10 boxes (4 Cherry, 4 Melon, 2 Mint). Each box includes: 100 unit-dose packets (0.5ml each); 100 applicator brushes



[www.3M.com/PreventiveCare](http://www.3M.com/PreventiveCare)



**3M Oral Care**  
2510 Conway Avenue  
St. Paul, MN 55144-1000 USA  
Phone 1-800-634-2249  
Web 3M.com/dental

**3M Canada**  
Post Office Box 5757  
London, Ontario N6A 4T1  
Canada  
Phone 1-888-363-3685

3M, ESPE and Vanish are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Printed in USA. © 3M 2017. All rights reserved. 70-2013-0865-0