

3M™ Clinpro™ Clear Fluoride Treatment – The next generation in fluoride treatment is here!

Effective fluoride uptake with a lower dosage.

With traditional fluoride varnishes, the patient's saliva must dissolve the sticky rosin-based coating to release the beneficial minerals, requiring longer treatment time. With Clinpro Clear Fluoride Treatment's innovative water-based formula, fluoride, calcium, and phosphate are ready to release upon contact with tooth surfaces, so teeth quickly receive the 2.1% dose of fluoride.

Not only do teeth get the same amount of fluoride uptake as with a 5% rosin-based fluoride varnish - Clinpro Clear Fluoride Treatment achieves these results in only 15 minutes.

Fast and easy application.

The unique L-Pop delivery system eliminates the need to peel or tear open packaging and is simply activated by pressing the package and delivering the product into the attached dispensing well. The L-Pop includes a single-use applicator brush that allows the product to be applied quickly and easily to any tooth surfaces. The water-based formula can be applied directly from the L-Pop well or be dispensed onto the back of a gloved hand.

- Apply with swiping motion — no need to brush onto individual tooth surfaces
- Migrates to hard to reach areas
- Quick and easy to apply unit dose

Ready for more patients to say “yes” to fluoride treatment?

In a clinical study, patients and professionals preferred Clinpro Clear Fluoride Treatment over tray gel fluoride and rosin-based varnish treatment.

- Patients rated Clinpro Clear Fluoride Treatment **96.3 out of 100 in appearance on their teeth.**
- **93.8% of patients** judged Clinpro Clear Fluoride Treatment to have a **smooth and slick mouthfeel.**
- Dental professionals rated their patients' **perception of their experience** with Clinpro Clear Fluoride Treatment **98 out of 100.**
- A new, water-based formula that delivers an improved experience for patients and dental professionals— with a minimum contact time of just 15 minutes.

Getting your patients back to enjoying life faster can promote patient acceptance of treatment, which can lead to better outcomes.

Source: Internal. Solventum data on file.

Extending protection to your orthodontic treatment plan.

Patients undergoing orthodontic treatment with traditional brackets or clear tray aligners may use Clinpro Clear Fluoride Treatment. Laboratory testing demonstrates that Clinpro Clear Fluoride Treatment is safe and compatible to use throughout all types of orthodontic treatment.

- Compatible with orthodontic brackets and bonding systems
- Compatible with clear tray aligners

And what about those suction lines?

Due to the water-based formulation of Clinpro Clear Fluoride Treatment, it is easy to clean from unwanted surfaces and will not clog suction lines. Clinpro Clear Fluoride Treatment can be easily wiped from unwanted surfaces with a damp cloth and washes completely from scrubs and other fabrics.

Fancy giving Clinpro Clear Fluoride Treatment a try? Why not request a FREE sample using the link below;

[Go.solventum.com/CCS](https://go.solventum.com/CCS)

Available in **watermelon, mint, or flavourless** options.