Solventum

3M[™] Clinpro[™] Clear Fluoride Treatment

L-Pop Dose Tips & Tricks

Safe to use entire single-unit dose



Easy to apply – just press and go!



Suction-line friendly formula

Choose your dispensing method:

Press into the well and go

Bend the well to fit into uour hand

Dispense onto the back of your gloved hand

Fluoride. Simplified.

Learn more at go.Solventum.com/ClinproClear.au



Press the triangle, dispense into well.

and remove the

3M Health Care is now Solventum

Bendable brush head helps you access hard-to-reach areas



Formula is free of common allergens*

*See allergen disclosure at go.Solventum.com/ClinproClear.au



Patients can eat or drink anything after a required contact time of just 15 minutes



Generally applied twice per year, but product can be applied more frequently if needed



Apply with a swiping motion - no need to brush onto individual tooth surfaces

















3M™ Clinpro™ Clear Fluoride Treatment

Item #	Product Information
	3M™ Clinpro™ Clear Fluoride Treatment 50 Pack
	50 unit-dose packages (0.50 ml L-Pop)
7200FF	Flavourless
7200M	Mint Flavour
7200W	Watermelon Flavour
	3M™ Clinpro™ Clear Fluoride Treatment 100 Pack
	100 unit-dose packages (0.50 ml L-Pop)
7210FF	Flavourless
7210M	Mint Flavour
7210W	Watermelon Flavour

3M™ Clinpro™ Take Home Treatments

Product Information
3M™ Clinpro™ 5000 1.1% Soldium Fluoride
Anti-Cavity Toothpaste [^] (113g tube)
Vanilla Mint Flavour
3M [™] Clinpro [™] Tooth Creme 0.21% Sodium Fluoride Anti-Cavity Paste with Tri-Calcium Phosphate (113g tube) Vanilla Mint Flavour

Request

a demo



KCI Medical Australia Pty Ltd

Level 3, Building A, 1 Rivett Road, North Ryde NSW, AU 2113

This product is intended for professional use only. Please always read the product instructions for use (IFU). ©Solventum 2025. Solventum, the S logo and Clinpro are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. Other trademarks are the property of their respective owners.

[^]Clinpro 5000 is a Pharmacist Only Medicine.