


One Bulk Fill Restorative


Class I restorations
Restaurations de classe I
Restauraciones de clase I

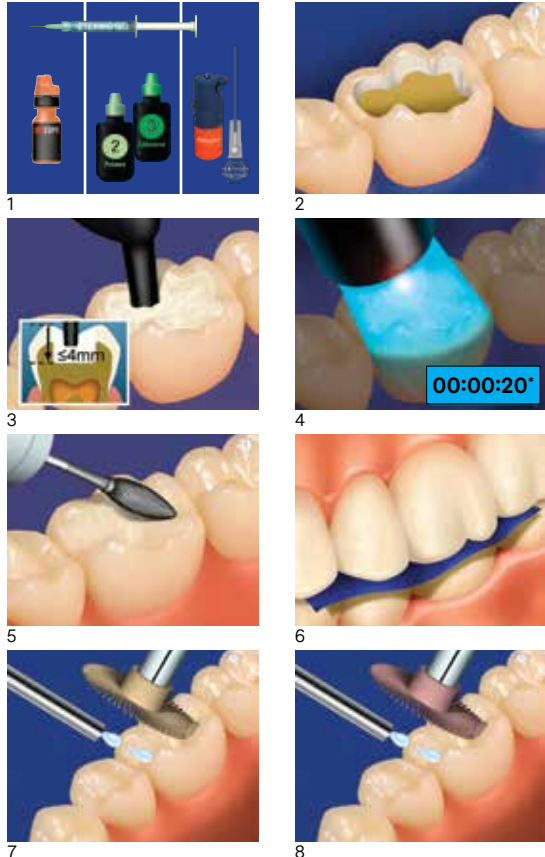

1 of 6

One Bulk Fill Restorative


Class II restorations
Restaurations de classe II
Restauraciones de clase II

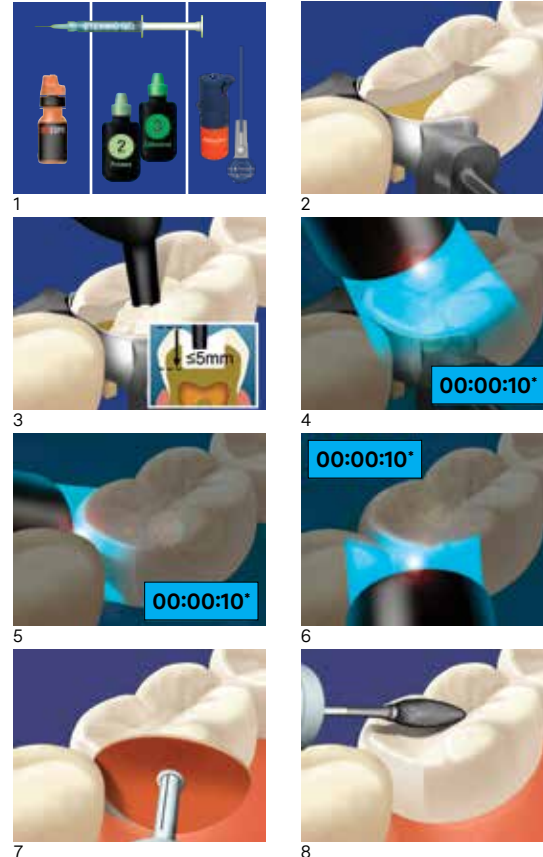

2 of 6

One Bulk Fill Restorative


Class II restorations
Restaurations de classe II
Restauraciones de clase II

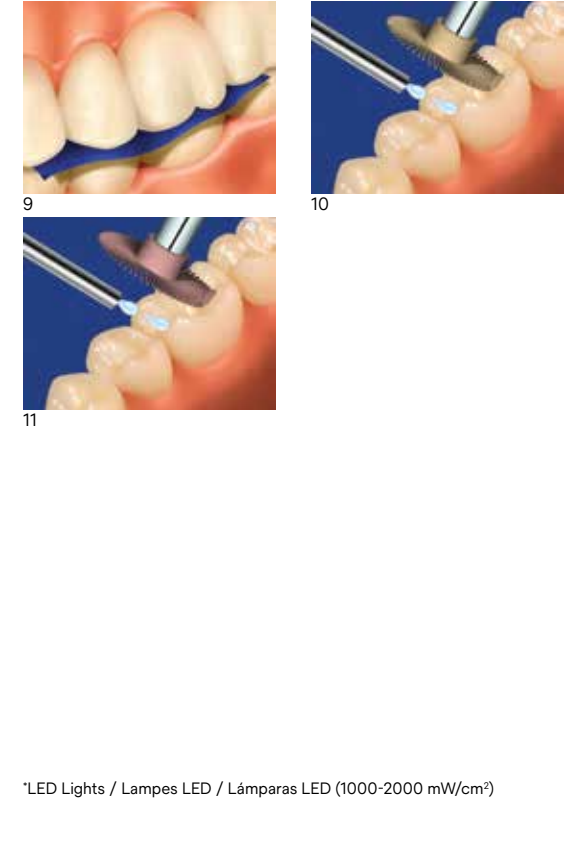


*LED Lights / Lampes LED / Lámparas LED (1000-2000 mW/cm²)

3 of 6

Anterior restorations
Restaurations antérieures
Restauraciones anteriores

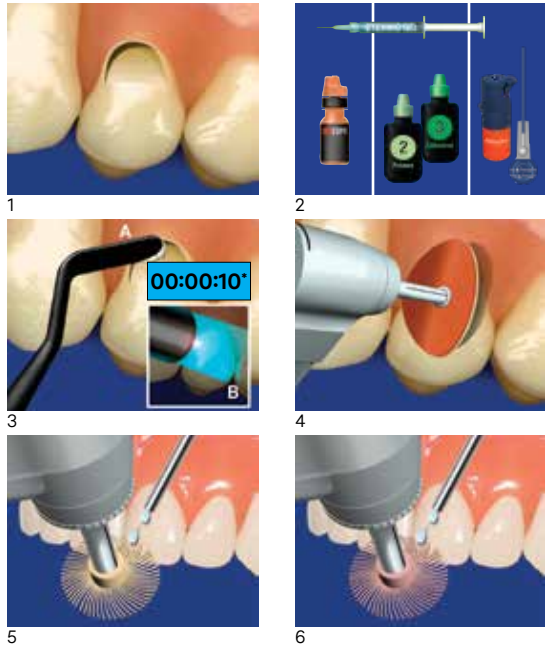

Core build up
Reconstitutions coronaires
Reconstrucciones de muñones

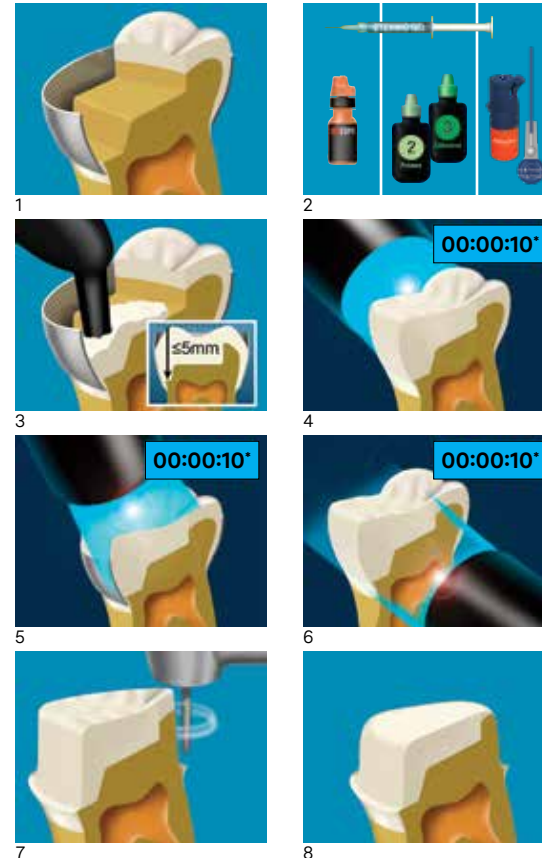

Restoration Type Type de restauration Tipo de restauración	Increment Depth Épaisseur de la couche Profundidad del incremento	Halogen Lights Lampes halogènes Lámparas halógenas (550-1000 mW/cm ²)	LED Lights Lampes LED Lámparas LED (1000-2000 mW/cm ²)
Class I	≤4 mm	40 sec.	20 sec.
Class II	≤5 mm	20 sec. Occlusal 20 sec. Buccal 20 sec. Lingual	10 sec. Occlusal 10 sec. Buccal 10 sec. Lingual
Anterior	≤3 mm	20 sec.	10 sec.
Core Build-Up	≤5 mm	20 sec. Occlusal 20 sec. Buccal 20 sec. Lingual	10 sec. Occlusal 10 sec. Buccal 10 sec. Lingual

*LED Lights / Lampes LED / Lámparas LED (1000-2000 mW/cm²)

