

3M™ Tube Securement Device


This strong, easy-to-use solution secures non-critical medical tubes. The tube securement device meets the Infusion Therapy Standards of Practice guidelines as an adhesive securement device and should be used in conjunction with sutures and/or primary securement dressings. The non-sensitizing silicone adhesive is gentle to skin and designed to support medical adhesive-related skin injury (MARS) and pressure injury prevention practices.

Uniquely shaped

Designed to lift tubes off skin and help promote consistent application.

Sterile device

In single-use packaging.

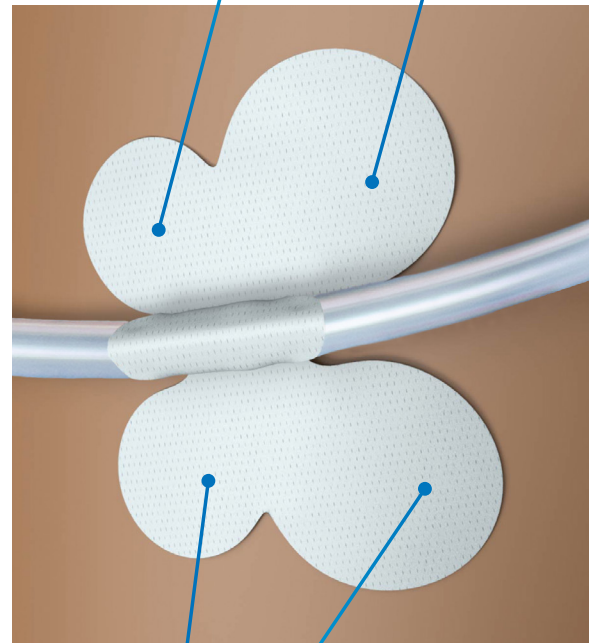

Reinforced soft cloth backing

Provides extra strength and flexibility.

Two sizes

Small device for medical tubes and catheters 6–12 Fr and large device for medical tubes and catheters 12–24 Fr.



Skin performance: The feather designates products that deliver the securement power you need while minimizing damage to skin.

Reliable securement


Unique adhesive secures medical tubes and catheters made from silicone, PVC and silicone coated latex for up to 4 days.¹

Ease of use

Easy to apply and removes cleanly in one piece; allows for standardization of technique.

Skin safe

Non-sensitizing silicone adhesive has low potential to cause dermatitis with minimal skin damage and pain upon removal.¹

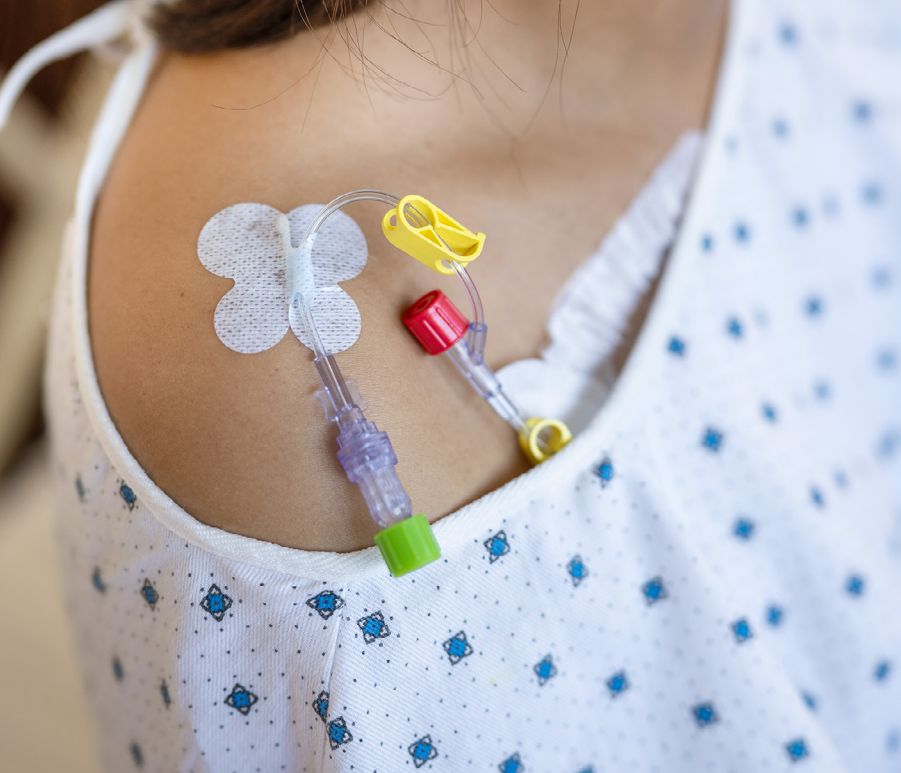

Indications and Applications

When it comes to securing IV tubing, there are a variety of factors to keep in mind, including standardization, securement performance, skin protection and clinician ease of use. The Tube Securement Device is used in conjunction with sutures and/or primary securement dressings and is indicated for non-critical tube securement. Two sizes accommodate tubes spanning 6-24 french.

Consider using the securement device as secondary securement for primary tubing and extension tubing on central lines and peripheral lines. It can help organize lines with catheters that have multiple lumens.

It is also indicated for securing urinary catheters and surgical drains.



Urinary catheters (in conjunction with urinary catheter balloons)



Surgical drains (in conjunction with sutures)

Feel secure in your choice. Trust the science behind 3M™ Tube Securement Device for reliable performance.

To order, contact your 3M Health Care Sales Representative or call the 3M Health Care Customer Helpline at 1-800-228-3957. Visit 3M.com/TubeSecurement for more information.



Refer to product Instructions for Use (IFU) for other important information.

Ordering Information

Product	Product Name	Product Number	Tube & Catheter Size	Product Size	Devices/Box	Boxes/Case
	3M™ Tube Securement Device, Small	1500U	6-12 French	1.7 in x 2.4 in (4,3 cm x 6,1 cm)	25	4
	3M™ Tube Securement Device, Large	1501U	12-24 French	2.2 in x 3.5 in (5,5 cm x 9,0 cm)	25	4



3M Company

2510 Conway Avenue
St. Paul, MN 55144-1000 USA

Phone 1-800-228-3957
Web 3M.com/Medical

© 2023 3M. All rights reserved.
3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited.
3M marks used under license in Canada.
All other marks are property of their respective owners.
GL 70-2013-1618-2