

Skin performance:
The feather designates products that deliver the securement power you need while minimizing damage to skin.

3M™ Nasogastric Securement Device


Application and Removal Guide

3M™ Nasogastric Securement Device Application




Prep the skin

- The skin and tubing should be clean, dry and free of residue and skin oil.
- To ensure good adhesion, allow all preps and skin protectants to dry completely.
- Open package and set aside.

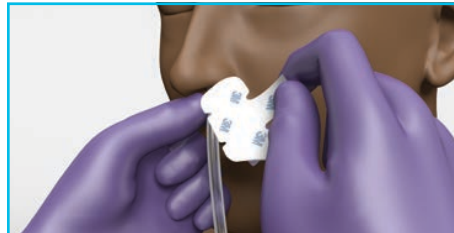


Remove device top liner

- Following facility protocol, insert the tube.
 - Remove the upper liner from the securement device to expose the adhesive.
- Note:** Keep tube stabilized throughout duration of securement.

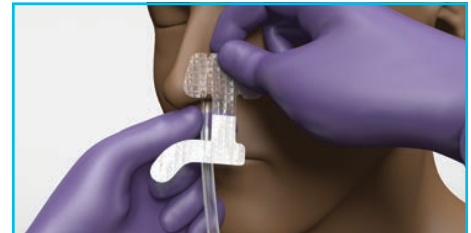


Place device

- Place the device on the side of the nose where tube is inserted.
- Center over the naris and place device onto nose.
- If able, ensure the device is fitted to conform to the crevice in the nose.
- Apply firm pressure on the device to enhance adhesion to skin.

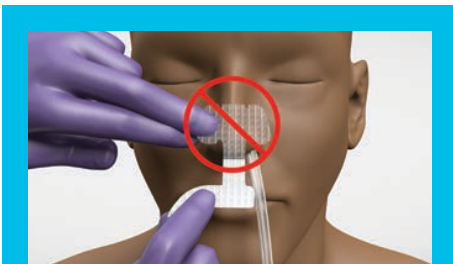


NOTE: Do NOT center over nose

Off-center placement helps the tube to hang freely to help minimize contact with the naris.

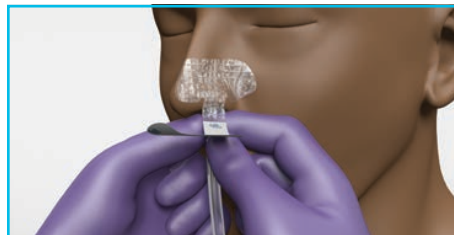

Remove bottom liner


- Remove the bottom liner from the securement device to expose the adhesive.
- Place tube in alignment with the vertical tail of the securement device.
- Apply firm pressure on the device to enhance adhesion to tubing.

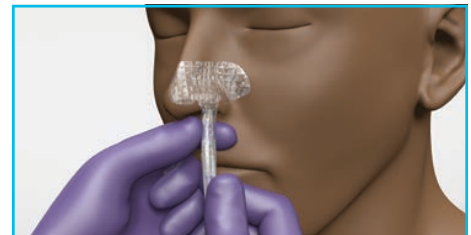

Wrap and secure


- Wrap the horizontal part of the tail in a spiral fashion around the tube.
 - Apply firm pressure to the device to enhance adhesion.
- Note:** Do not wrap the tail of the 3M™ Nasogastric Securement Device too tightly or closely to the naris as this could lead to medical device-related pressure injury.

3M™ Nasogastric Securement Device Maintenance and Tips

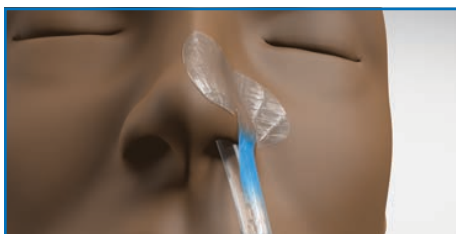

Non-adhesive features

- A non-adhesive section at the base of the nose is intended to help the tube hang freely.
- Do not tape over in attempt to aid in securement of the product.
- A non-adhesive tab at the end of the tail is intended to help aid in removal.

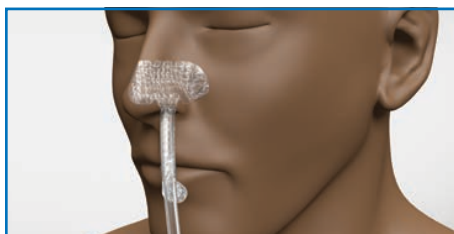

Monitor


- The site should be observed regularly for signs of skin complications or tube migration per facility protocol.
- Skin complications may be signaled by redness, color changes, skin that is abnormally warm or cool to the touch, non-blanching skin on palpation, pain, tingling, or itching.
- If skin irritation or injury is suspected, remove the device, inspect the site directly, and determine appropriate medical intervention.

Device Replacement



Unwrap

- While stabilizing the tube, use the non-adhesive tab at the end of the tail to unwrap the device from the tube.

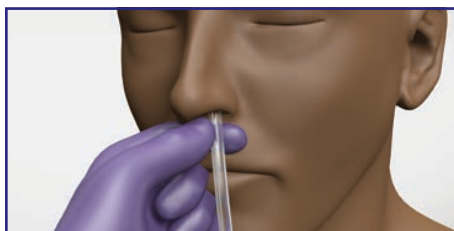

Remove and replace

- Remove the adhesive device from the nose while stabilizing the tube.
- Once the device is removed, a new one may be applied.

Note: Peel low and slow.

Note: Ensure the skin and tube are clean, dry, and free of residue and skin oil before reapplication.

Device Removal



Discontinuing NG therapy

- Remove the securement device from the nose.
- Remove the tube from the patient.

Note: Peel low and slow.

Refer to product Instructions for Use for other important information.



Scan to view a product training video or visit [3M.com/NasogastricSecurement](https://www.3m.com/NasogastricSecurement) to learn more.



3M Company
2510 Conway Avenue
St. Paul, MN 55144-1000 USA

Phone 1-800-228-3957
Web [3M.com/Medical](https://www.3m.com/Medical)

© 2022 3M. All rights reserved.
3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. Used under license in Canada.
US_70-2013-1311-4