




3M™ Nasogastric Securement Device

Application and removal guide

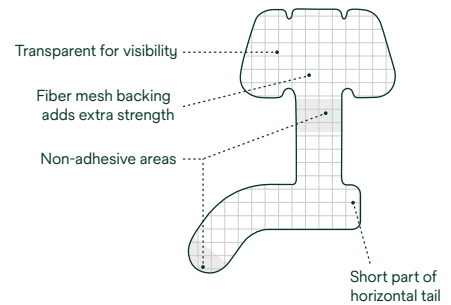

Skin performance:

The feather designates products that deliver the securement power you need while minimizing damage to skin.

3M™ Nasogastric Securement Device application




1. Prep the Skin

- Thoroughly clean the nose and tubing with an alcohol wipe or cleanser of facility's choosing.
- The skin and tubing should be clean, dry and free of residue and skin oil.

Note: A clean starting surface is critical to successful adhesion to skin and to tubing.


- Open package and set aside.

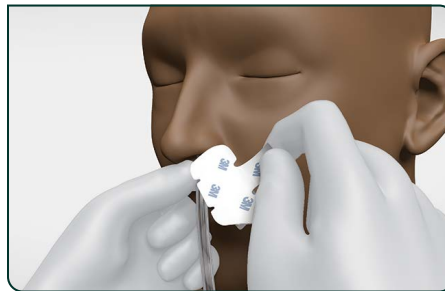

2. Remove device top liner


- Following facility protocol, insert the tube.
- Remove the upper liner from the securement device to expose the adhesive.

Note: Keep tube stabilized throughout duration of securement.

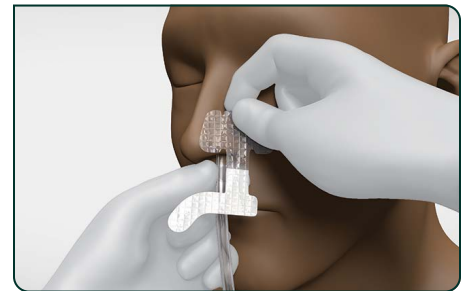

3. Place device


- Place the device on the side of the nose where tube is inserted.
- Center over the naris and place device onto nose.
- Ensure the device is fitted to conform to the crevice in the nose.
- Apply firm pressure on the device to enhance adhesion to skin.

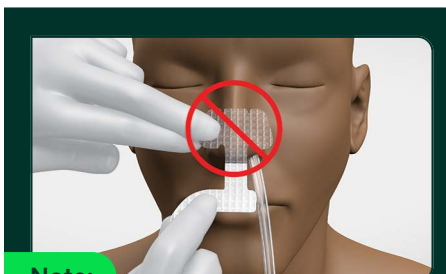

Note:

Do NOT center over nose

Off-center placement helps the tube to hang freely to help minimize contact with the naris.

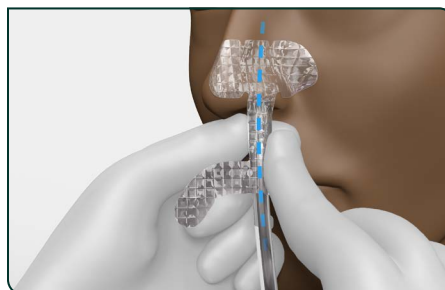

4. Remove bottom liner and begin wrap

- Remove the bottom liner to expose the adhesive.
- Place tube in alignment with the vertical tail of the securement device.
- Apply firm pressure on the device to enhance adhesion to tubing.
- Wrap the short tab (on right) of the horizontal part of the device around the tube before continuing.




5. Continue wrap and secure

- Wrap the remaining horizontal part of the tail in a spiral fashion around the tube.
- Apply firm pressure to the device to enhance adhesion.
- A non-adhesive tab at the end of the tail is intended to help aid in removal.


Note: Do not wrap the tail of the 3M™ Nasogastric Securement Device too tightly or closely to the naris as this could lead to medical device-related pressure injury.

3M™ Nasogastric Securement Device maintenance and tips

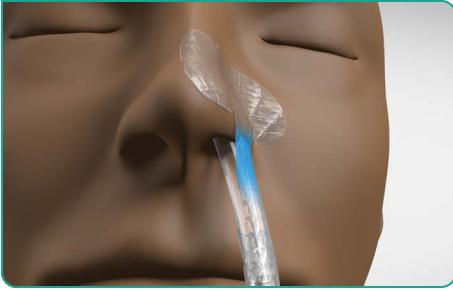

Non-adhesive feature

- A non-adhesive section at the base of the nose is intended to help the tube hang freely.
- Do not attempt to tape over the non-adhesive section of the device.

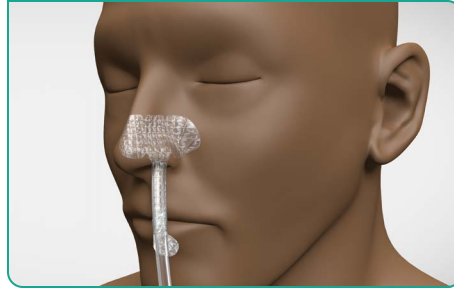

Monitor

- The site should be observed regularly for signs of skin complications or tube migration per facility protocol.
- Skin complications may be signaled by redness, color changes, skin that is abnormally warm or cool to the touch, non-blanching skin on palpation, pain, tingling, or itching.
- If skin irritation or injury is suspected, remove the device, inspect the site directly, and determine appropriate medical intervention.

Device replacement



1. Unwrap

- While stabilizing the tube, use the non-adhesive tab at the end of the tail to unwrap the device from the tube.



2. Remove and replace

- Remove the adhesive device from the nose while stabilizing the tube.
- Once the device is removed, a new one may be applied.

Note: Peel low and slow while removing device from nose.

Note: Ensure the skin and tube are clean, dry, and free of residue and skin oil before application of a new securement device.

Device removal



Discontinuing NG therapy

- Remove the securement device from the nose.
- Remove the tube from the patient.

Note: Peel low and slow.



Refer to product
Instructions for Use for other
important information.



Solventum Corporation
2510 Conway Avenue
St. Paul, MN 55144-1000
USA

Phone 800-228-3957
Web Solventum.com



Scan the QR code to view a product training video, or visit
3M.com/NasogastricSecurement
to learn more.

© 2025 3M. All rights reserved.

3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited.
Used under license in Canada.

US_70-2013-1311-4