

Skin performance:
The feather designates products that deliver the securement power you need while minimizing damage to skin.

3M™ Tube Securement Device

Application and Removal Guide

3M™ Tube Securement Device Application



Initiate application


- Open package.
- Completely remove the device from the liner.



Apply first wing


- While minimizing tube manipulation, apply one wing (half) of the device onto the patient's skin.

Note: Do not stretch device during application as tension may result in skin trauma.

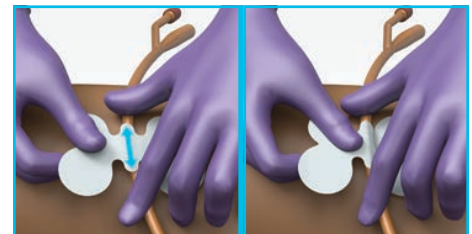

Secure tubing

- Align the tube along the center “body” of the device.
- Fully wrap the device around the tube.

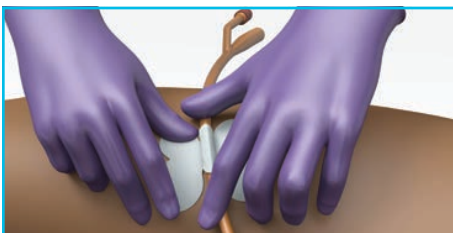


Apply second wing

- Apply the other wing (half) of the device onto the patient's skin.
- Apply firm pressure on the device, including the edges, to enhance the adhesion to the skin and to the tubing.




Check final positioning

- Ensure the tube is lifted slightly from the skin to reduce the risk of pressure injury.

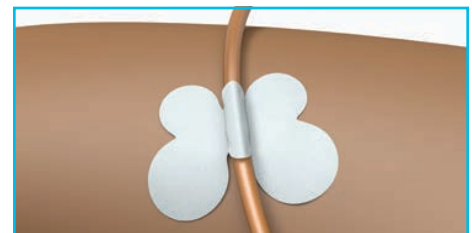


Applied device

3M™ Tube Securement Device Replacement or Removal

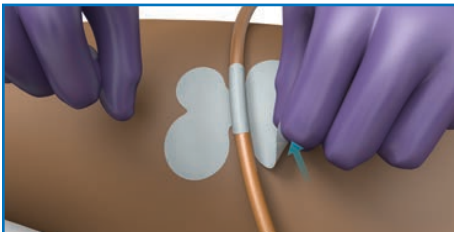

Initiate removal

- Minimize tube manipulation during removal.
- Peel back one corner of the device.

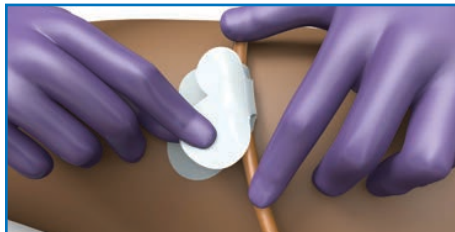

Remove first wing

- Stabilize the tube to the patient.
- Keep device parallel to skin.
- Completely remove one wing.

Note: Peel low and slow.



Stabilize and remove

- Carefully peel the device off the tube.
- Remove the remaining wing of the device from the skin.

Note: Peel low and slow.

Refer to product Instructions for Use for other important information.



Scan to view a product training video or visit [3M.com/TubeSecurement](https://www.3m.com/TubeSecurement) to learn more.



3M Company
2510 Conway Avenue
St. Paul, MN 55144-1000 USA

Phone 1-800-228-3957
Web [3M.com/Medical](https://www.3m.com/Medical)

© 2022 3M. All rights reserved.
3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. Used under license in Canada.
US_70-2013-1330-4