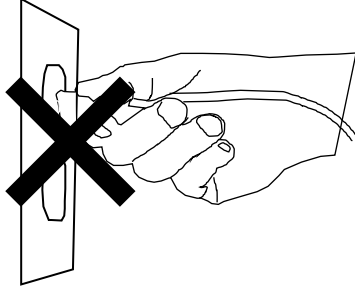
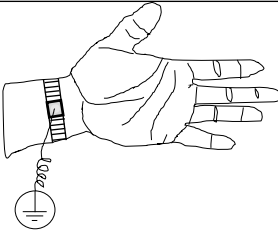
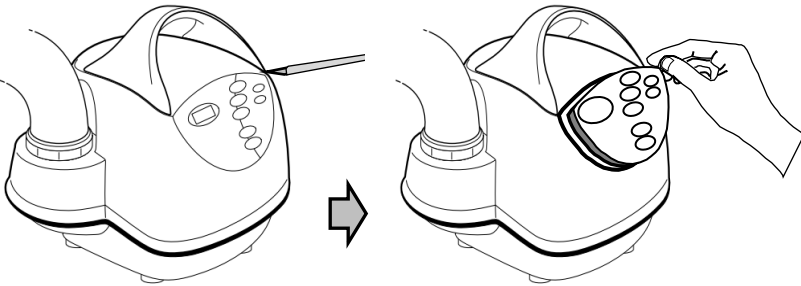
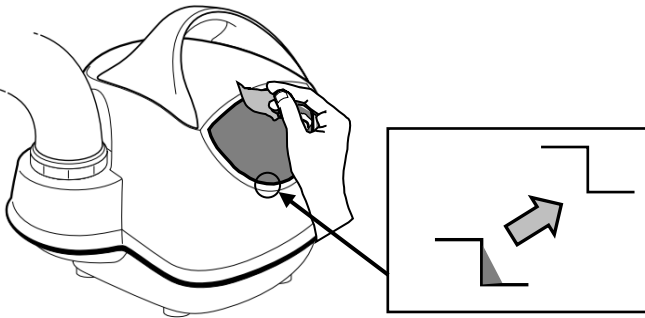
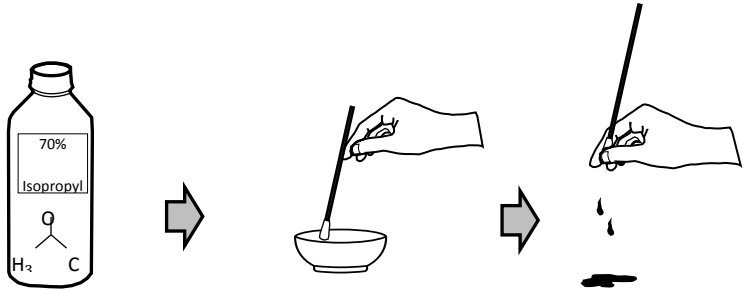
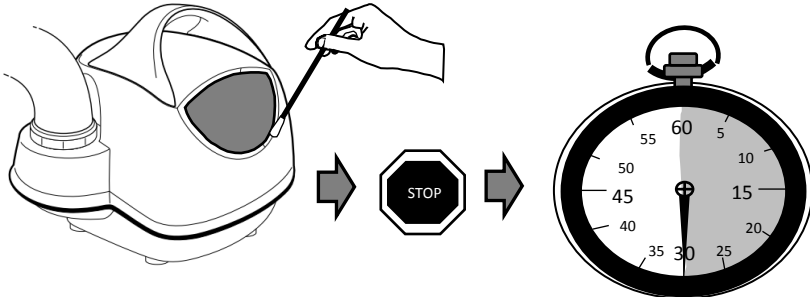
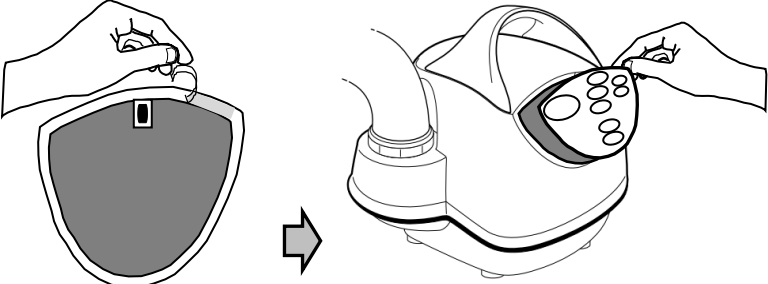
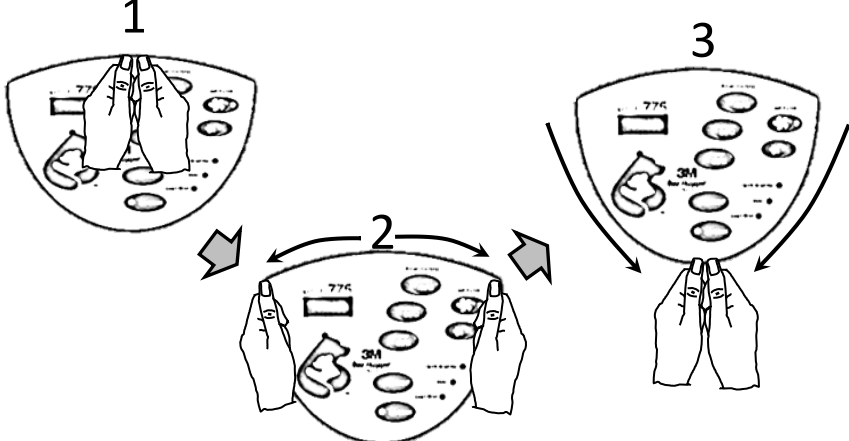


Replacing the User Interface Label on 3M™ Bair Hugger™ warming unit, Models 675, 775*, 875 or the 3M™ Ranger™ Pressure Infusor, Model 145

*For warming unit, model 775 with serial number 188120 and above the device will need to be sent to service for a label change.

The basic steps outlined below should be followed for all User Interface labels. Warming unit model 775 is used to demonstrate steps.

<p>Tools/equipment needed for user interface label replacement</p>	<ul style="list-style-type: none"> • Precision knife • Ground strap • Cotton tipped applicator • 70% Isopropyl Alcohol • Soft, clean, dry cloth
<p>Step 1 Disconnect unit from power source.</p>	
<p>Step 2 Use ground strap to control static discharge.</p>	
<p>Step 3 Use precision knife to carefully lift corner and remove user interface label. Try not to gouge plastic on enclosure.</p>	
<p>Step 4 Remove any adhesive residue or debris that may be left on surface and recessed edge by rubbing with soft, clean, dry cloth.</p>	

<p>Step 5 Dip the cotton tipped applicator in 70% isopropyl alcohol, then remove excess liquid.</p>	
<p>Step 6 Use the cotton tipped applicator to clean outer edge of plastic surface. Wait 30 seconds for alcohol to fully evaporate from surface.</p>	
<p>Step 7 Peel off backing liner and apply user interface label to the device enclosure.</p>	
<p>Step 8 Apply pressure to adhesive area on the user interface label in order to adhere the user interface label to the device enclosure.</p>	

Discard removed user interface label according to institutional protocol.

For additional support

USA: 1-800-228-3957

Outside USA: Contact your local 3M representative

3M Company

2510 Conway Ave.
St. Paul, MN 55144 USA

Phone 1-800-228-3957

Web 3M.com/Medical

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. Used under license in Canada.

Issue Date: 2021-12