

Reminders for 3M™ V.A.C.® Therapy with 3M™ V.A.C.® Peel and Place Dressing

Indications

V.A.C.® Peel and Place Dressing can be used on:

- Chronic wounds
- Acute wounds
- Traumatic wounds
- Subacute wounds
- Dehisced wounds
- Partial-thickness burns
- Ulcers (such as diabetic, pressure or venous insufficiency)
- Flaps
- Grafts

Warnings

Do not use on wounds with:

- Undermining greater than 2 cm
- Tunneling
- Depth greater than 6 cm



Dressing should cover the entire wound bed and extend onto intact skin.

Find the right size

Small - EZ5SML	Medium - EZ5MED	Large - EZ5LRG
Max wound size: 6.1 cm x 8.6 cm	Max wound size: 11.1 cm x 16.6 cm	Max wound size: 13.6 cm x 24.2 cm
Max Depth: 2 cm	Max Depth: 4 cm	Max Depth: 6 cm



Always read and follow the detailed Instructions for Use along with important safety information provided in the packaging.

Application tips for 3M™ V.A.C.® Therapy with 3M™ V.A.C.® Peel and Place Dressing



Wear time

Dressing can remain in place for **up to 7 days**.¹



Device settings

Set the device to **-75 to -150 mmHg** on **continuous mode**.

Do not

- Cut foam.
- If skin barrier/protectant is used under the adhesive portion of the dressing, do not reposition the dressing as adhesive properties may be reduced.
- Use with other V.A.C.® Therapy Dressing Kits or foams, as they have a shorter wear time.
- Push foam portion of the dressing down. Allow the dressing to draw down and conform the wound bed as negative pressure is applied.
- Stretch dressing.

Drape Tips

- Apply loosely over the wound area.
- Leave at least a 5 cm border of drape wherever possible.
- Can reposition within 20 minutes of initial application.
- For curved anatomy, slits may be cut in the drape portion of the dressing to help reduce overlap, drape folds or remove wrinkles.

Watch
how-to-
apply
video



Customer service:
800-275-4524

Clinical support:
800-275-4524 x 56650

Technical support:
800-275-4524 x 3

NOTE: Specific indications, contraindications, warnings, precautions, and safety information exist for these products and therapies. Please consult a clinician and product Instructions for Use prior to application. Rx only.

¹ SAT-BSER-05-869347 VAC Peel and Place (Ganymede) BSER. 510(k) K222859.

© Solventum 2024. US-EN 70-2013-1691-9