

# Reminders for Solventum™ V.A.C.® Therapy with Solventum™ V.A.C.® Peel and Place Dressing

## Indications

V.A.C.® Peel and Place Dressing can be used on:

- Chronic wounds
- Acute wounds
- Traumatic wounds
- Subacute wounds
- Dehisced wounds
- Partial-thickness burns
- Ulcers (such as diabetic, pressure or venous insufficiency)
- Flaps
- Grafts

## Warnings

Do not use on wounds with:

- Undermining greater than 2 cm
- Tunneling
- Depth greater than 6 cm



Dressing should cover the entire wound bed and extend onto intact skin.

## Find the right size

| Small - EZ5SML                            | Medium - EZ5MED                             | Large - EZ5LRG                              |
|-------------------------------------------|---------------------------------------------|---------------------------------------------|
| <b>Max wound size:</b><br>6.1 cm x 8.6 cm | <b>Max wound size:</b><br>11.1 cm x 16.6 cm | <b>Max wound size:</b><br>13.6 cm x 24.2 cm |
| <b>Max Depth:</b><br>2 cm                 | <b>Max Depth:</b><br>4 cm                   | <b>Max Depth:</b><br>6 cm                   |



Always read and follow the detailed Instructions for Use along with important safety information provided in the packaging.

## Application tips for Solventum™ V.A.C.® Therapy with Solventum™ V.A.C.® Peel and Place Dressing



### Wear time

Dressing can remain in place for **up to 7 days**.<sup>1</sup>



### Device settings

Set the device to **-75 to -150 mmHg** on **continuous mode**.

#### Do not

- Cut foam.
- If skin barrier/protectant is used under the adhesive portion of the dressing, do not reposition the dressing as adhesive properties may be reduced.
- Use with other V.A.C.® Therapy Dressing Kits or foams, as they have a shorter wear time.
- Push foam portion of the dressing down. Allow the dressing to draw down and conform the wound bed as negative pressure is applied.
- Stretch dressing.

#### Drape Tips

- Apply loosely over the wound area.
- Leave at least a 5 cm border of drape wherever possible.
- Can reposition within 20 minutes of initial application.
- For curved anatomy, slits may be cut in the drape portion of the dressing to help reduce overlap, drape folds or remove wrinkles.

Watch  
how-to-  
apply  
video



Customer service, including technical/clinical support:  
1-800-275-4524

NOTE: Specific indications, contraindications, warnings, precautions, and safety information exist for these products and therapies. Please consult a clinician and product Instructions for Use prior to application. Rx only.

<sup>1</sup>SAT-BSER-05-869347 VAC Peel and Place BSER. 510(k) K222859.

© Solventum 2025. US-EN 70-2013-1691-9