



# 3M™ V.A.C.® Ultra 4 Therapy Unit with 3M™ V.A.C.® Therapy

## V.A.C.® Ultra 4 Therapy Unit





## Setting up the therapy unit

### 1. Attach the canister



V.A.C.® Therapy: 500ml or 1000ml canister (acute care only).

### 2. Connect the dressing tubing (after applying the dressing) Align, push, and twist

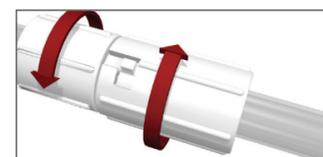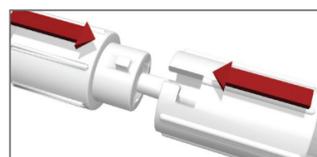


**Note:** Refer to the instructions for use for complete details regarding dressing application.

## Initiating V.A.C.® Therapy

Press and hold power button until light comes on.




### Choose therapy screen

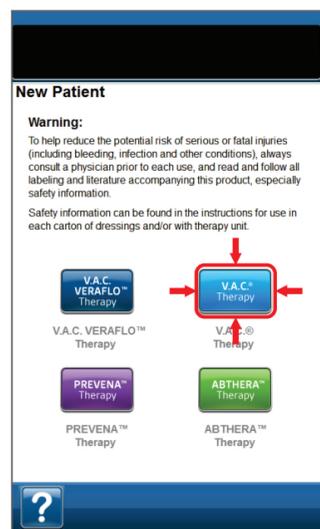

Select V.A.C.® Therapy

### Confirm settings screen

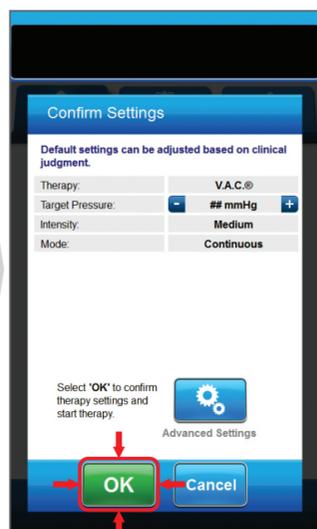

Select OK to accept default settings

### 3M™ Seal Check™ Feature



Drawdown begins

### Home screen V.A.C.® Therapy




V.A.C.® Therapy begins

## Enhanced features

### Pre-leak indicator early detection system


V.A.C. Therapy Home screen.

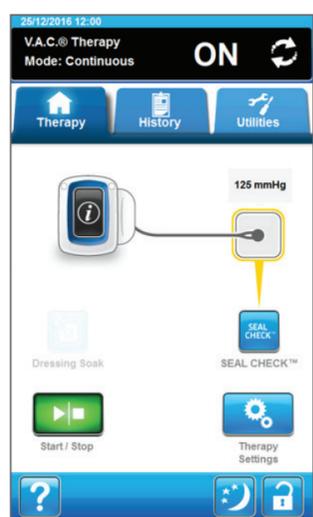

The yellow box around the dressing notifies the user that a leak is present and an alarm will occur soon if not addressed.

### 3M™ Dynamic Pressure Control™ Therapy

V.A.C.® Therapy advanced settings screen.

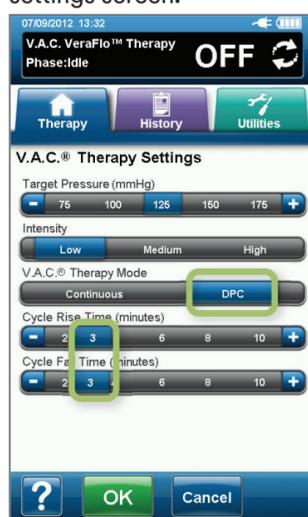

Configure V.A.C. Therapy as prescribed and select OK.

## Contraindications

Do not place foam dressings of the V.A.C.® Ultra Therapy System (including both V.A.C. Therapy and 3M™ V.A.C. Veraflo™ Therapy Dressings) directly in contact with exposed blood vessels, anastomotic sites, organs, or nerves.

**V.A.C.® Therapy is contraindicated for patients with:**

- Malignancy in the wound.
- Untreated osteomyelitis.
- Non-enteric and unexplored fistulas.
- Necrotic tissue with eschar present (Note: After debridement of necrotic tissue and complete removal of eschar, V.A.C.® Therapy may be used.)
- Sensitivity to silver (3M™ V.A.C.® Granufoam Silver™ Dressing only)

**Note:** Refer to the warnings and precautions section of the safety information sheet for additional information including details on potential risks associated with bleeding and infection. For complete information on indications, contraindications, and other safety information please reference the user manual and safety information sheet.



**Note:** If an alarm condition cannot be resolved, contact a healthcare provider or local representative.

**Note:** Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. 70-2013-1106-8 PRA-PM-AT-00061 (09/21)



**Veraflo™**  
Therapy

# 3M™ V.A.C.® Ultra 4 Therapy Unit with 3M™ Veraflo™ Therapy

## Setting up the therapy unit

- Press and hold the power button until light comes on.**
- Once the startup screen is visible, attach the 3M™ V.A.C. Veralink™ Cassette**
- Extend the solution container hanger arm**
- Spike and hang the solution container bag/bottle**
- Connect the dressing tubing (after applying the dressing)**  
Align, push, and twist.
- Attach the 500ml or 1000ml canister**

## V.A.C.® Ultra 4 Therapy Unit



### Contraindications

Do not place foam dressings of the V.A.C.® Ultra Therapy System (including both V.A.C. Therapy and V.A.C. Veraflo Therapy Dressings) directly in contact with exposed blood vessels, anastomotic sites, organs, or nerves.

V.A.C. Therapy and Veraflo Therapy are contraindicated for patients with:

- Malignancy in the wound.
- Untreated osteomyelitis.
- Non-enteric and unexplored fistulas.
- Necrotic tissue with eschar present (Note: After debridement of necrotic tissue and complete removal of eschar, V.A.C.® Therapy may be used.)
- Sensitivity to silver (3M™ V.A.C.® Granufoam Silver™ Dressing only).

**Note:** Refer to the warnings and precautions section of the safety information sheet for additional information including details on potential risks associated with bleeding and infection. For complete information on indications, contraindications, and other safety information please reference the user manual and safety information sheet.

**Additional contraindications specific to Veraflo Therapy:**


- Do not use V.A.C. Dressings with Octenisept®, hydrogen peroxide or solutions that are alcohol-based or contain alcohol.
- Do not deliver fluids to the thoracic or abdominal cavity due to the potential risk to alter core body temperature and the potential for fluid retention within the cavity.
- Do not use Veraflo Therapy unless the wound has been thoroughly explored due to the potential for inadvertent instillation of topical wound solutions to adjacent body cavities.

Connect the installation line and then the V.A.C. Therapy line.

**Note:** Refer to the Instructions for Use for complete details regarding Dressing Application.

## Initiating Veraflo Therapy

### Start up screen

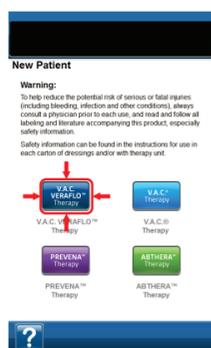


Select Veraflo Therapy.

### Confirm settings



Select OK to accept default settings.

### 3M™ Seal Check™ Feature

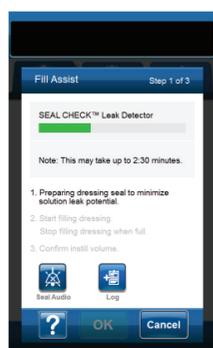


Drawdown begins.

### Fill assist screen



Select start/stop on the fill assist screen to begin delivering solution to the wound.

### Fill assist screen

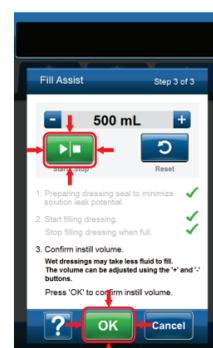

Stop fill assist by depressing start/stop. Select OK when satisfied with instillation volume.

### Home screen Veraflo™ Therapy



Veraflo Therapy begins

## Enhanced features

### Veraflo Therapy – advanced settings overview (optional)



Select advanced settings.



Adjust pressure, intensity and therapy mode. Select OK to accept settings.




Turn fill assist off, select start phase. Adjust instill volume, soak time and V.A.C.® Therapy time.


### Information screens – Veraflo Therapy



Select i button from the home screen to continue to the therapy summary tab.

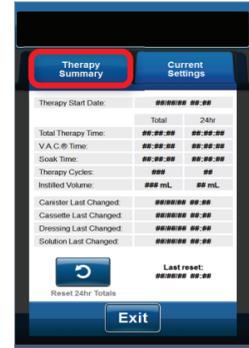

Use the therapy summary tab to review a cumulative summary of therapy provided.

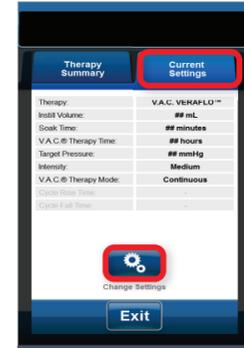

Use current settings tab to review the current therapy settings. Select change settings to continue to the confirm settings screen.

**Note:** If an alarm condition cannot be resolved, contact a healthcare provider or local representative.

**Note:** Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. 70-2013-1106-8 PRA-PM-AT-00061 (10/21)

